



Clarity Viewer User Manual



| Image Capture Review

Contents

Copyright and Proprietary Notice.....	5
Contact and Product Information.....	7
Safety Information.....	9
iCRco Warranty.....	11
Revision History.....	15
 Chapter 1: Introduction.....	 17
Overview.....	17
 Chapter 2: Setup and Configuration.....	 19
Local System Configuration.....	19
Updates.....	19
Launching Clarity Viewer.....	19
Logging In.....	20
Preferences.....	22
Connection.....	22
Monitor.....	23
International.....	24
Annotations.....	25
Tools.....	28
Reporting.....	29
License Information.....	32
Server.....	32
Hanging Protocols.....	33
Prefetch.....	33
Advanced.....	35
Archive Storage Limit.....	38
Warning Lights.....	38
Logging Out & Quitting.....	39
 Chapter 3: Operation and Usage.....	 41
Workflow Controls.....	41
Search.....	42
Worklist.....	44
Image Layout and Navigation.....	45
Image Manipulation Functions.....	48
Cine Tools.....	50
Rotate Tools.....	52
Measurements.....	53
Annotations.....	56
Filters.....	57
Reporting Tools.....	58
Advanced Tools.....	61
Pelvic Toolset.....	61
Other Tools.....	65
Exporting Studies.....	70
Saving and Sending a Study.....	71

Clarity Viewer Hotkeys.....	72
Chapter 4: Mammography.....	75
Mammography Features.....	75
Hanging Protocols.....	76
Creating and Editing a Hanging Protocol.....	77
Information Tab.....	78
Layout Tab.....	79
Chapter 5: DICOM Printing.....	83
Advanced DICOM Printer.....	83
Printing to a Paper Printer.....	88
DICOM Print Troubleshooting for Windows.....	93
Configuring a Printer with the QPC Print Tool.....	94
Clearing the Java Cache.....	95
Setting the Display Back to Default Using Regedit.....	97

Copyright and Proprietary Notice

Proprietary Notice and Disclaimer

The information herein disclosed is the property of iCRco., Inc. Information in this document is subject to change without notice and does not represent a commitment by iCRco to incorporate changes or improvements in units previously sold or shipped. No part of this document may be reproduced or transmitted in any form, electronic or mechanical, including photocopying and recording, for any purpose other than the purchaser's own use without the express written permission of iCRco.

Copyright

Copyright© iCRco Inc. 2003-2015. All rights reserved.

Trademark

iCR 1000™, iCR 1000 Dual™, iCR 2600™, iCR 2600 Dual™, iCR 2600SF™, iCR Vet™, iCR Vet Dual™, iCR Mobile™, iCR 1-D™, iCR Chiro™, iCR Chiro Dual™, iCR 3600™, iCR 3600SF™, iCR 3600+™, iCR 3600M™, iCR VERTX™, iDR™, Fusion DCR™, Chrome™, Chrome Duo™, ChromeM™, Chrome+™, ChromeNDT™, Air DR™, Air DRc™, Fusion DR Mobile™, QPC XSCAN32™, Captera™, XC™, iSCAN™, and www.icrco.com are the trademarks of iCRco. All other trademarks are the property of their respective owners, and are hereby acknowledged.

Terms

Any one of the following iCR products will be referred to as "CR unit" throughout this document: iCR Vet™, iCR Vet Dual™, iCR Mobile™, iCR 1-D™, iCR Chiro™, iCR Chiro Dual™, iCR 3600™, iCR 3600SF™, iCR 3600M™, iCR 3600+™, iCR VERTX™, iCR Fusion™, Chrome™, ChromeM™, Chrome+™, ChromeNDT™, and Chrome Duo™.

Any one of the following iCR products will be referred to as "DR unit" throughout this document: Air DR™, Air DRc™, and Fusion DR Mobile™.

Any one of the following iCR products will be referred to as "iCR desktop unit" throughout this document: iCR VERTX™. The iDR-17™, iDR-34™ and iDR-51™ will be referred to as the "iDR unit" throughout this document.

Contact and Product Information

Contact Information

iCRco., Inc.	Americas
Address:	26 Coromar Drive, Goleta, CA 93117
Phone:	+1.310.921.9559
Fax:	+1.805.685.1308
Email:	info@icrcompany.com
Web:	http://www.icrco.com

iCRco., Inc.	Europe
Address:	Paderborner Str. 21, 44143 Dortmund, Germany
Phone:	+49.231.92733669

Safety Information



Read and understand the installation and operating instructions before starting the software.

	<div>EC</div> <div>REP</div>	
US Headquarters iCRco, Inc. 26 Coromar Drive Goleta, CA 93117 USA 310-921-9559	European Office iCRco Europe GmbH Paderborner Str. 21 44143 Dortmund Germany +49-231-92733669	

Conventions



Warning:

A WARNING indicates a potentially hazardous situation which, if not avoided, could result in death or serious injury.



Caution:

A CAUTION indicates a potentially hazardous situation which, if not avoided, may result in minor or moderate injury. It may also be used to alert against unsafe practices.



Note:

A NOTE indicates important information that helps you better utilize your CR Unit and Software.



Important:

An IMPORTANT indicates either potential damage to hardware or loss of data and tells you how to avoid the problem.

Malfunctioning Equipment

If any iCRco software product shows signs of malfunction, try restarting the computer; if the problem persists or is reproducible discontinue the use of the product immediately and contact Technical Support at 310-921-9559.

Mammography

Access to Clarity Viewer's mammography feature is controlled by the USB Dongle Key.

Clarity PACS

Clarity Viewer is a component of the 510(k) cleared Clarity PACS (K002911).

Indications for Use

Clarity Viewer is software that retrieves digital images and data from various sources (e.g. CT scanners, MR scanners, ultrasound systems, RIF Units, computed & direct radiographic devices, secondary capture devices, scanners, imaging gateways, or other imaging sources).

Images (including mammography) and data can be retrieved and displayed within the system and/or across computer networks at distributed locations.

Clarity's archiving engine utilizes lossless algorithms for compression services (JLSL, J2KR, and JPLL). Lossless compressed mammographic images can be reviewed for primary image interpretation. Lossy compressed mammographic images received from outside sources and digitized film screen images must not be reviewed for primary image interpretation.

Mammographic images may only be interpreted using an FDA cleared monitor that offers at least 5 MP resolution and meets other technical specifications reviewed and accepted by FDA.

iCRco Warranty

iCRco, Inc. ("iCRco") values your business and always strives to provide high-quality products and services. All iCRco products are provided with an initial warranty so that the hardware and software are covered from the date of purchase. This limited warranty solely applies to new products manufactured by or for iCRco and originally purchased from iCRco or an authorized dealer of iCRco products for your own use. In addition, an extended warranty is available for most new and recently purchased iCRco products for an additional charge.

Hardware Limited Warranty

iCRco warrants its hardware products to be free of defects in materials and workmanship for a period of one (1) year from the date of installation, which must be within 60 days of the original shipment from iCRco subject to the limitations set forth herein. If a product proves to be defective in material or workmanship during the warranty period, iCRco will, at its sole option, repair or replace the product with a similar product. Repaired and replacement products may be or include refurbished or remanufactured parts. Any replacement item assumes the remaining warranty period of the original product. iCRco provides no warranty for any third party hardware or software included with any product or later acquired.

Software Limited Warranty/Support

iCRco warrants that its QPC XSCAN32, Captera, XC, iSCAN, and/or Clarity PACS software originally provided with any product will substantially conform to iCRco's specifications and that the media, not including hard drives, on which the software is furnished will be free from defects in materials and workmanship under normal use for a period of one (1) year from the date of installation, which must be within 60 days of the original shipment from iCRco. iCRco's sole obligation under this warranty is limited to making reasonable efforts to ensure such conformity and to supply the consumer with a corrected version of the software as soon as it is practical after the consumer has notified iCRco of any nonconformity. iCRco does not warrant that the operation of any software will be uninterrupted, glitch- or error-free, or that functions contained in the software will operate in the combinations which may be selected for use by the user or meet the user's requirements. This limited software warranty will be void if the software is modified without the written approval of iCRco or is used outside of the recommended parameters or equipment. iCRco does not provide any warranty or support for any other software.

iCRco agrees to provide one (1) year of telephonic and/or e-mail-based support for QPC XSCAN32, Captera, XC, iSCAN, and/or Clarity PACS software originally provided with any new iCRco Air DR product from the date of installation, which must be within 60 days of the original shipment from iCRco. All software support shall be limited to making reasonable efforts to resolve iCRco software issues and shall be limited to iCRco's regular business hours. In addition, iCRco will provide revisions and upgrades to its software upon request (when available) during the first year after the software was originally shipped from the iCRco factory. The initial support period will include support via remote login software, only if the customer has access to the Internet from that PC and only if the customer agrees iCRco shall have no liability in connection with its support efforts. Remote login software allows iCRco technical support to remotely access the customer's PC via the Internet for the purposes of rendering technical support. Please note that this warranty, including software support, does not include computer hardware, third party software, or operating system or network issues, which are outside the control of iCRco.

Warranty Product Technical Requirements

iCRco requires that all DR, CR, Scanner, and/or products requiring PCs be fitted and installed with a 1300VA (780W) uninterruptible power supply ("UPS"). iCRco recommends the APC 1300 specification UPS or equivalent. For warranty evaluation and service, iCRco requires the customer to provide an Internet connection (DSL or Dial-up) or the minimum of a phone line accessible by an extension cord to the product enabling iCRco technicians to perform remote diagnostics on installed equipment. In addition, each iCRco product must be installed, maintained, and operated in accordance with the respective product manual. Failure to comply with these requirements will result in a voided warranty claim.



Warning: No modifications to this equipment are allowed.

Requesting Warranty Service

For information on obtaining warranty service, call iCRco's customer support at (310) 921-9559 or (866) 907-7704. In order to evaluate a warranty service request, iCRco requires the following information: the iCRco serial number of the product, a detailed description of the problem, customer name and contact information, product location and operating conditions, a copy of the purchase documents, and sufficient information and authorization, including a liability release as to any loss of data (that should always be backed up), software or network injury, or downtime, allowing iCRco technicians remote access to the product. Product may not be returned to iCRco without first obtaining a Return Material Authorization ("RMA") number from iCRco. Prior to providing an RMA, iCRco may require remote access to the product. If iCRco determines that the product may be defective, is under warranty, and necessitates a return to iCRco for service, an RMA number and instructions for return of the product will be given. iCRco is not responsible for any unauthorized returned product, i.e. one for which an RMA number has not been issued by iCRco.

Warranty service requires all authorized returns be shipped to the iCRco factory prepaid and insured. All such authorized returns are the customer's responsibility. For products sold and located within the United States, iCRco will pay for return shipping.

Products being returned are only to be shipped in iCRco approved shipping containers. The original box and packaging materials are approved and should be kept for moving and/or shipping the product. Approved packaging may also be purchased from iCRco for an additional charge. iCRco shall have no liability nor responsibility for warranty service to any product that is not shipped in an iCRco approved shipping container, that is damaged from incorrect packaging, or that is damaged during shipping.

Additional Warranty Limitations and Extent of Warranty

This warranty does not apply if the product has been damaged by accident, misuse, or abuse. In addition, warranty service does not include the repair of failures or defects caused by: unauthorized attachments to any iCRco product, unsuitable physical or operating environment, maintenance or repair by anyone other than iCRco or the iCRco authorized dealer that sold the product, operation of a product beyond its duty cycle, use of the product outside of its specifications, the use of supplies, parts, materials, software, or interfaces not furnished, authorized, or recommended by iCRco. If the product, including any software has been opened, tampered with, modified, or altered in any way without written authorization by iCRco, the warranty will no longer apply.

This warranty applies only to products manufactured by, or for, iCRco; products can be identified by an "iCRco" serial number as originally affixed to the product. Any modification to the iCRco serial number tag or its attachment to the product shall immediately void the warranty.

This warranty is non-transferable and subsequent owners must contact iCRco to establish if the equipment is eligible for an extended warranty.

THERE ARE NO WARRANTIES, EXPRESS OR IMPLIED, WITH RESPECT TO ANY iCRco PRODUCT OTHER THAN AS SPECIFICALLY SET FORTH HEREIN, AND iCRco SPECIFICALLY DISCLAIMS ANY IMPLIED WARRANTIES OR CONDITIONS OF MERCHANT ABILITY, FITNESS FOR A PARTICULAR PURPOSE, AND SATISFACTORY QUALITY. ANY WARRANTIES THAT MAY NOT BE DISCLAIMED UNDER APPLICABLE LAW ARE LIMITED IN DURATION TO THE INITIAL WARRANTY PERIOD AND NO WARRANTIES, EXPRESS OR IMPLIED, WILL APPLY AFTER THIS PERIOD. ALL INFORMATION, SPECIFICATIONS, PRICES, AND SERVICES ARE SUBJECT TO CHANGE AT ANY TIME WITHOUT NOTICE.

Limitation of Remedies and Liability/Exclusion of Damages

The exclusive remedy for any defective product is limited to the repair or replacement of the defective product. iCRco shall have a reasonable time after determining that a defective product exists to repair or replace a defective product. iCRco's entire liability for any product is limited to the actual purchase price for the product. This limitation applies even if iCRco cannot or does not repair or replace any defective product.

IN NO EVENT WILL iCRco BE LIABLE FOR ANY GENERAL, SPECIAL, CONSEQUENTIAL, OR INCIDENTAL DAMAGES, including but not limited to, damages related to the loss of use, loss of recorded product, the installation of replacement product, or any inspection, testing, or redesign caused by defect or by the repair or replacement of any product arising from a defect in any product. This exclusion of damages applies even if the customer advises iCRco or an iCRco dealer of the possibility of such damages. This limitation of remedies also applies to claims against any suppliers or dealers of iCRco. iCRco and its suppliers' and dealers' limitations of remedies are not cumulative. Such suppliers and dealers are intended beneficiaries of this limitation. iCRco is not liable for any claim by or against the customer arising from a third party claim.

Revision History

Revision	Date	Approved by	Comments
A	2009-04-06	MS	Initial release
B	2010-03-03	MS	Updated with 1.3 features
C	2010-07-08	MS	Added CE and FDA approval label
D	2010-10-05	MS	Updated with 1.4 features
E	2011-03-09	MS	Updated minimum computer requirements
F	2013-05-23	KS	Updated for 4.8 Release
F-02	2013-07-30	KS	Frontmatter Update
G	2013-07-31	KS	Updated for 4.10 Release
H	2013-12-20	KS	Feature Clarification
I	2014-01-10	KS	Updated for 4.12 Release
J	2014-08-26	KS	Updated for 4.14 Release
11	2015-05-15	SS	Updated for 4.14 Release

Chapter 1

Introduction

Overview

Clarity is an easy to maintain, scalable, digital image, and information management solution. Clarity provides a secure, HIPAA-compliant image management and display for medical imaging modalities including: Computed Radiography (CR), Magnetic Resonance (MR), Computed Tomography (CT), and Ultrasound (US). The Clarity Archive provides DICOM services including DICOM storage, DICOM routing, and DICOM Modality Worklist for patient information systems. The Clarity Archive also includes integration with most existing RIS and HIS systems. The Clarity Archive supports the HL-7 protocol allowing integration with many EMR systems.

The Clarity Viewer is a multi-modality image viewer designed to provide rapid and easy access to medical images. Multiple monitors are automatically detected and easily configured. An integrated Worklist provides flexible search options such as filtering by patient ID, accession number, study ID, modality, study status, and many more. The Clarity Viewer provides robust and easy-to-use image manipulation and annotation tools including support for CR, DR, and cross-sectional imaging (e.g., US, CT & MR modalities).

Clarity is approved by the FDA and may be utilized for mammography purposes.



Read and understand the installation and operating instructions before starting the software.

Chapter

2

Setup and Configuration

Local System Configuration

Minimum Viewing Workstation Configuration:

CPU: 2 GHz Core2Duo

System RAM: 4 GB (6 GB for non-CR diagnostic review)

Hard Drive: 200 GB

Graphics Card: 1280 x 1024/32-bit color capable graphics card

Monitor: High-contrast, diagnostic-quality monitor (two monitors recommended)

Operating System: Windows 7 Service Pack 1, 64-Bit

Java Version: JAVA 8

Domain Configuration:



Note:

The User may access Clarity Viewer using a domain name, but the network must be properly configured. Please contact your system administrator to assign a domain name to the Clarity Viewer. Any Workstations wishing to access the Clarity Viewer via a domain name must be configured on the same Windows domain as the Archive. Domain misconfiguration may result in a non-functioning Clarity Viewer.

Firewall Configuration:

Workstations wishing to use the Clarity Viewer or DICOM Send and Query/Retrieve to the Clarity Archive need to have TCP ports 104 (DICOM), 8080 (Clarity launch page non-SSL port), 8443 (Clarity launch page SSL port), and 11112 (Clarity Archive port) and UDP port 137 (data transfer) open in the Windows firewall and any third party hard and/or software firewalls running on the network.

Updates

When a new version of the Clarity Viewer has been deployed to the Clarity server, the Clarity Viewer software will automatically update itself from the Clarity server onto the Workstation on the next launch.

Launching Clarity Viewer

Initial Installation and Launch of Clarity Viewer

1. Launch the web browser.
2. In the address bar, enter `https://IPADDRESS:8443/icrco-viewer` where

IPADDRESS

is the IP address of the Clarity Archive.



Note:

It is recommended that the user register a domain name for the Clarity server and purchase an SSL certificate from a trusted Certificate Authority when using remote viewing. These actions help ensure secure data transmission.



Note:

If accessing Clarity through a secure local area network or VPN, the user can enter `http://IPADDRESS:8080/icrco-viewer` instead of the "https" address. This allows to user to avoid the browser security warning dialog.

3. Check that Java is installed on the workstation.



Note:

If Java needs to be installed, go to `http://www.java.com`. Download and install the free software.

4. Click the **Clarity.exe** link.
5. Save the Clarity.exe file. Save it or place it in a location that you would like to launch it from, such as the desktop.
6. Double click the Clarity.exe icon to start Clarity. The program will scan the network to locate the PACS server, download any necessary files, and then start the viewer.



Clarity.exe



Note:

When first launched and when the software is updated, Clarity will need to download multiple files. This might require a lengthy period of time.

Logging In

After launching, the Clarity Viewer will open and present a login dialog box. Enter a User name and password, then click **OK** to continue.



Note:

The default username and password is "user" and "user".


Note:

If you do not have a User name and password, please contact your System Administrator.

LDAP Authentication

If the Clarity Archive is set up with LDAP (Lightweight Directory Access Protocol), then check the LDAP Authentication box.

LDAP is used to allow user to have a single login account across multiple applications, commonly known as SSO (Single Sign-On). SSO integration with certain systems might require further configuration of Clarity. Contact iCRco for more information.

Clarity Server Selection

The login screen allows you to select which Clarity Archive you wish to access. This is useful if you have more than one Archive that you regularly use.

To add a new Archive to the list, click the drop-down arrow on the right side of the Server field. Select **Add Server**.

This will bring up the Add Server dialog box.

You may enter in the Clarity Archive information including server address and the name you want displayed in the Login panel. It is possible to set the new archive as the default server listed when the Login dialog appears. Simply check the default server box.

Click **OK**.



Note:


Upon logging in, the server you're currently connected to will be shown underneath the Clarity logo.



Preferences

The preferences can be accessed by clicking on the **Configuration** button, located in the upper, right-hand corner of the Worklist window. Clicking the **Configuration** button will cause the **Preferences** dialog to pop-up.

Connection

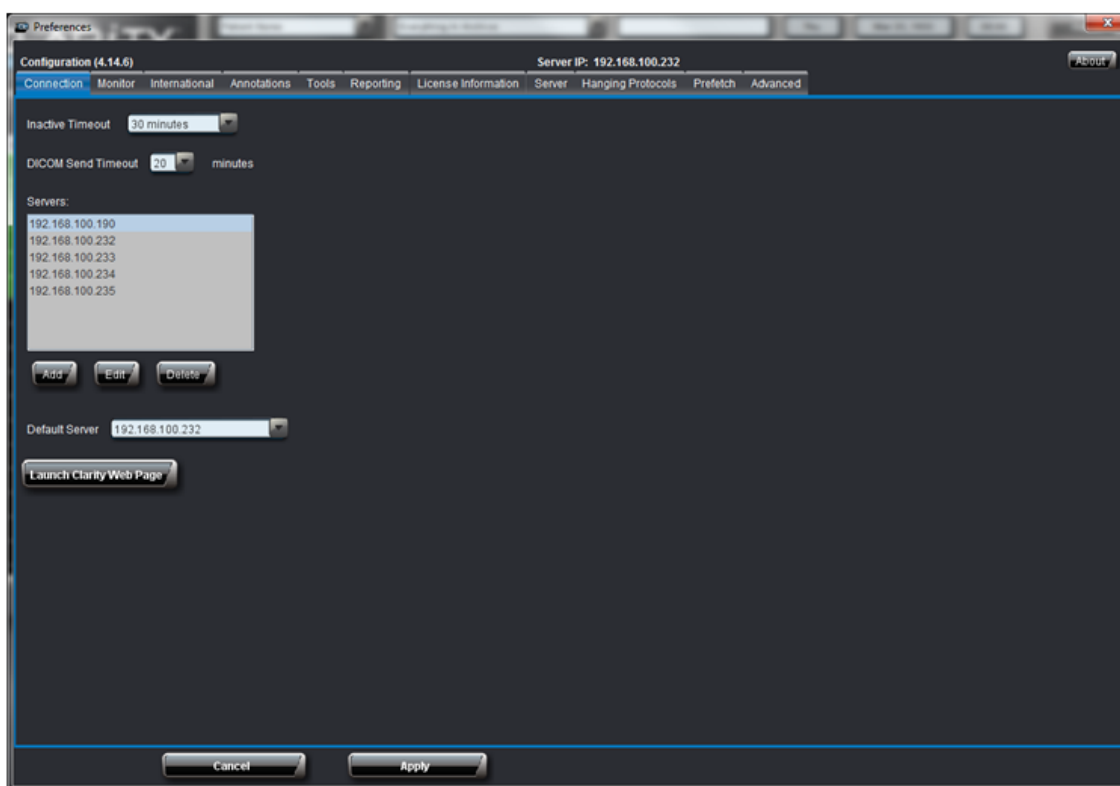

The Connection Tab in Preferences control the various settings related to digital communication between the Viewer and Archive. The user can control timeout and server selection in this tab.

Inactive Timeout

The Inactive Timeout preference allows the User to specify the length of time before the Clarity Viewer automatically logs out. To modify the Inactive Timeout preferences, click the Connection tab in the **Preferences** dialog. Select the desired length of time. The default duration is 20 minutes.

DICOM Send Timeout

The user can set how long Clarity PACS will attempt to DICOM Send a study to an Archive, before stopping due to timeout.

Servers Field

The Servers field lists the Clarity Archive servers currently registered on the view station. From this screen, a user can add, edit, or delete an archive from the list.

Default Server


The user can select which archive is the default server from the dropdown menu.

Launch Clarity Web Page

Clicking this button will take the user to the Clarity web page on the PACS server. This page contains links to archive administration, documentation, and other tools.

Monitor

The monitor preferences can be accessed by clicking on the **Configuration** button. This will bring up the **Preferences** dialog. From there, click the **Monitor** tab.

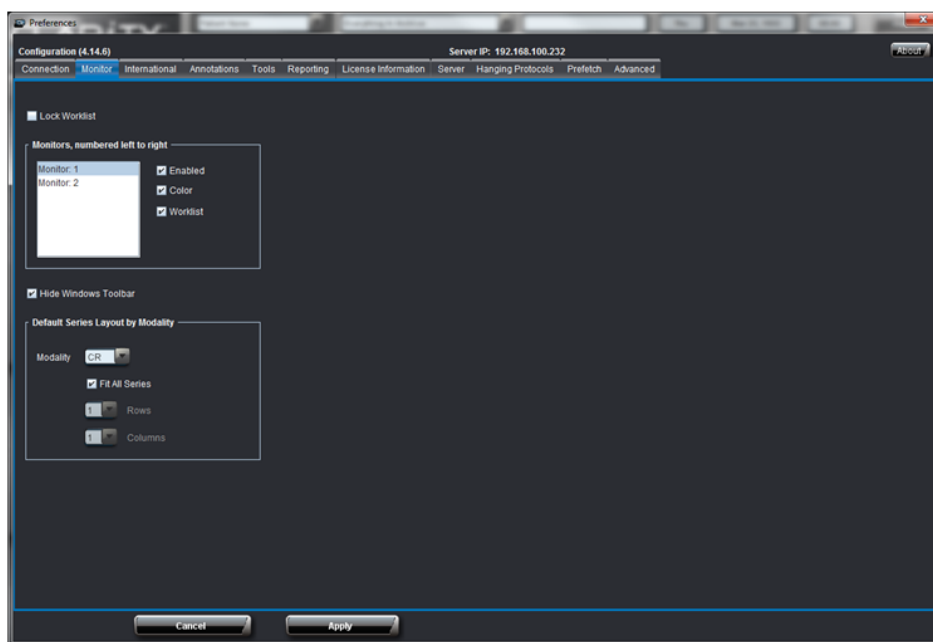

From the **Preferences** dialog, the User can configure each monitor attached to the computer. The monitors are numbered from left to right.



Note:

Changing monitor preferences will require a restart of the Clarity Viewer for the changes to take effect. The application will **not** restart on its own. The User will need to restart the application manually.

Lock Worklist

When selected, the worklist will always be displayed. This setting should be left off for workstations with a single monitor.

Monitor Configuration

The monitors in this list are numbered from left to right. For each monitor, the user may indicate whether it is enabled, color, and whether it should display the worklist. Since the worklist can only be displayed on one monitor, selecting it for one monitor will de-select it for other monitors. At least one monitor must be enabled, and at least one monitor must display the worklist. If these requirements are not met, a warning will be issued when the user hits apply.

Hide Windows Tool bar


When selected, Clarity will use the entire screen. When unselected, Clarity will leave room for the Windows toolbar.

Default Display Layout Per Monitor

The user can determine the default series layout for each modality. If "Fit All Series" is selected, Clarity will display all series for the study. Otherwise, the user may determine the number of rows and columns to display by default for each modality.

International

The International options can be accessed by clicking on the **Configuration** button. This will bring up the **Preferences** dialog. From there, click the **International** tab.

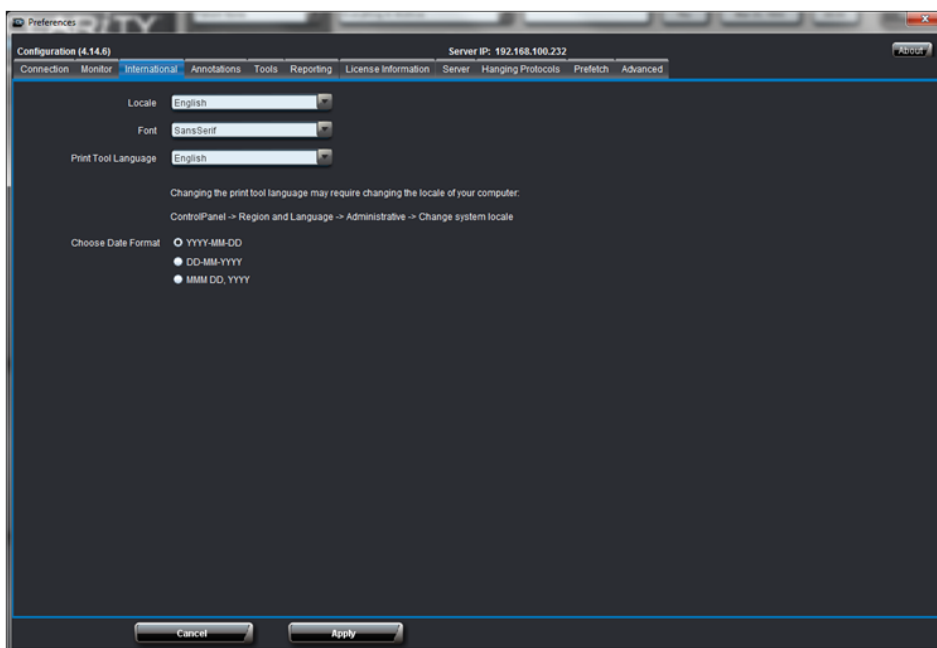

Locale

Setting the locale will set the language, text orientation (left to right or right to left), and date & time for the given region. Currently supported locales are: Chinese, English, German, Hindi, Japanese, Portuguese, Serbian, Spanish, and Thai.

Font

The font option allows the User to select the font face for the application. This includes the font face for the Worklist and all dialog boxes inside the Clarity Viewer application, but does not include annotation fonts which are set separately in the **Annotations** tab. See Section [Annotations](#) on page 56 for more information on setting Annotation fonts.

Print Tool Language

The user may choose which language the QPC Print Tool uses.




Note:

It may be necessary to change the computer's locale in order for QPC Print Tool to properly display the selected language.

Date Format

Setting the date format will update the study date on the wordlist and the image overlay to the format chosen from the date format options.

Annotations

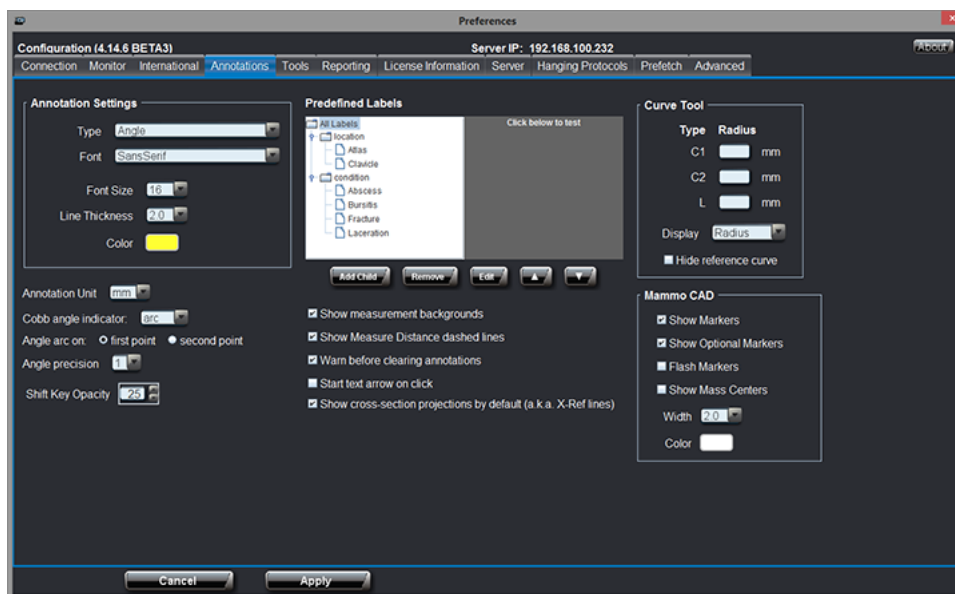

The Annotation Options allow the User to set the font, font color and size, annotation line thickness, and annotation color. The User may apply these options to all annotations at once (by selecting **All** from the **Annotation Type** drop-down menu), or the User may select the type of annotation, and set the parameters individually for each annotation type.

Annotation Type

Using the **Annotation Type** drop-down menu allows the User to select configuration parameters for each individual annotation tool, or set the parameters for all tools at once (select **All** in the **Annotation Type** drop-down menu).

Annotation Font Type

Allows the User to select any font installed on the operating system to display annotations using text.

Annotation Font Size

Allows the User to select the font size for annotations using text. The User may choose a font size between eight (8) and twenty (20) points.

Annotation Line Thickness

Allows the User to select a value between one-half (0.5) and four (4.0) for the thickness of the annotation lines.

Annotation Color

Allows the User to select the color for the annotation lines and text.

Click the color icon to open the **Color Chooser** dialog. The User may select from swatches (commonly used, pre-defined colors) by clicking on a swatch color, HSB (**H**ue, **S**aturation, and **B**rightness) by selecting a point somewhere in the colored square, or RGB (**R**ed, **G**reen, and **B**lue) by adjusting the sliders to the desired result.

Annotation Unit

The user may choose the unit of measurement.

Cobb angle indicator

The user may select whether the Cobb angle uses lines, an arc, or neither.

Angle arc

The user can choose which angle is measured when selecting three points. If the first point setting is chosen, then the angle will measure the angle where the first point is the vertex. If the second point setting is chosen, then the angle will measure the angle where the second point is the vertex.

Angle precision

The angle precision drop-down menu allows you to select how many digits past the decimal point to display in an angle measurement. For example, if an angle measured 141.59°, Clarity will display:

- 0 Digits - 142°
- 1 Digit - 141.6°
- 2 Digits - 141.59°

Shift Key Opacity

This value controls indicates how opaque annotations will be when the "shift" key is depressed. For example, a value of 25 indicates that the annotations will be 25% opaque, and 75% dimmed. If this value is set to zero, the annotations will be invisible when the "shift" key is depressed, and a value of 100 would indicate no dimming. See the section "Drawing Over Existing Annotations" for more information.

Predefined Labels

This tool allows you to set up predefined labels, which can be placed on an image using the Advanced Tools.

To add a label, select a folder and click the **Add Child** button. Type in the label text and click **OK**.



Note:

To create a new folder, click the **Add Child** button. Name the label. Then add a child to the label you just created. This will convert the label to a folder.

To remove a label or folder, select the item and click **Remove**.



Note:

Removing a folder will delete the folder and all labels within it.

To edit the name of a label or folder, select the item and click **Edit**.

It is possible to reorder the predefined label list by using the up and down buttons. Select the item you wish to move, then click either button to move it.



Note:

If you click within the grey testing area, you can see what the predefined label menu will look like.

**Note:**

Predefined Labels is part of the Advanced Tools. This setting will be greyed out if the Advanced Tools feature is not activated on the dongle license.

Show measurement backgrounds

When this box is checked, the measurements will be displayed in a small, white bubble field.

Show Measure Distance dashed lines

When checked, the measured distance will be indicated with a dashed line. When unchecked, only the measurement will be displayed.

Warn before clearing annotations

When this box is checked, Clarity will notify the user before annotations are removed.

Start text arrow on click

When this box is unchecked, drawing an arrow annotation begins with the end of the arrow. When the box is checked, drawing an arrow annotation begins with the arrow head.

Show cross-section projections by default

When this box is checked, the cross-reference lines will automatically appear when viewing images.

Curve Tool

The user can set the radius for the reference curves used in the Curve tool. The radius can be selected for the C1 - T1 Cervical Curve (C1), the C2 - C7 Cervical Curve (C2), and the Lumbar Curve (L). To set the radius, click the corresponding box and type in the desired radius in millimeters.

The Display drop-down box allows you to select whether you want the Curve Tool to display the curve's angle measurement, the curve's radius measurement, both measurements, or neither.

Hide Reference Curve: When selected, Hide Reference Curve will not show the fixed (dotted) line.

**Note:**

The Curve Tool is part of the Advanced Tools. This setting will be greyed out if the Advanced Tools feature is not activated on the dongle license.

Mammography CAD

The mammography features are only activated when the user has purchased the mammography package. The mammography CAD settings only apply to mammography studies processed by an MG CAD software.

- **Show Markers:** Show/Hide mammography CAD markers when Clarity starts.

**Note:**

Hitting the F4 key will toggle the visibility of the markers.

This setting toggles all markers including optional markers mentioned below.

- **Show Optional Markers:** Show/Hide optional mammography CAD markers when Clarity starts.

**Note:**


Hitting the F5 key will toggle the visibility of the optional markers.

- **Flash Markers:** When selected, MG CAD markers will flash when the image is loaded.
- **Show Mass Centers:** If this option is selected, Clarity will display the center of each mass.
- **Width:** Set the pixel width of the MG CAD markers.

- **Color:** Set the color of the MG CAD markers.

Tools

The Tools options can be accessed by clicking on the **Configuration** button. This will bring up the **Preferences** dialog. From there, click the **Tools** tab.

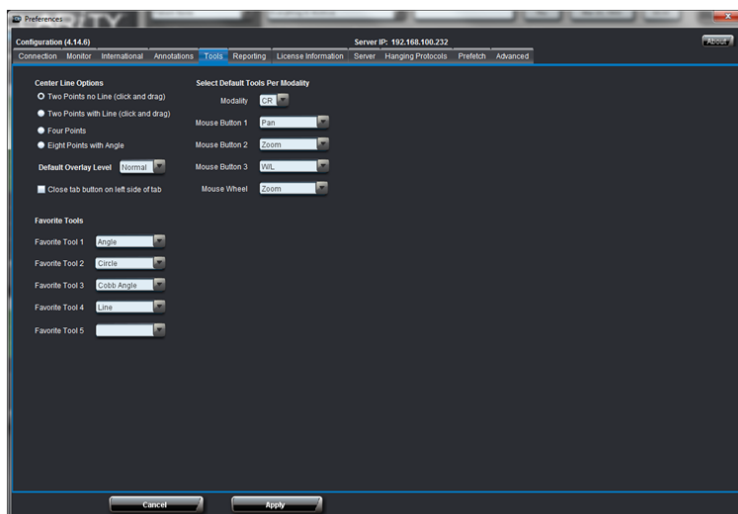

Center Line Options

The Center Line tool has four different drawing options. The options are: Two Points without Line, Two Points with Line, Four Points, or Eight Points with Angle.

Default Overlay Level

The user may select the amount of information displayed in the corner of the image.

- None - No text is displayed.
- Normal - Minimal text is displayed.
- Full - A large amount of text is displayed.

Close tab button on left side of tab

If this box is checked, the close button "X" will be relocated to the left side of the tab.

Select Default Tools Per Modality

Selecting the default tools per modality allows the User to override the default mouse button mapping described in Section [Default Mouse Actions](#) on page 46. First select the desired modality, then choose the desired mouse button mapping. For example, for the CT modality, the user may want to map Pan to the left mouse button, map Zoom to the middle mouse button, map W/L to the right mouse button, and map Zoom again to the mouse wheel.

Favorite Tools

It is possible to choose which tools appear in the Favorite Tools section of the toolbar. There are five available slots in the toolbar. Use the dropdown menu to select the desired tool. After the configuration has been applied, the selected tools will appear in the toolbar.



Note:


This feature was formerly known as Frequently Used Tools.

Favorite Tools



Reporting

The Reporting preferences can be accessed by clicking on the **Configuration** button. This will bring up the **Preferences** dialog. From there, click the **Reporting** tab.

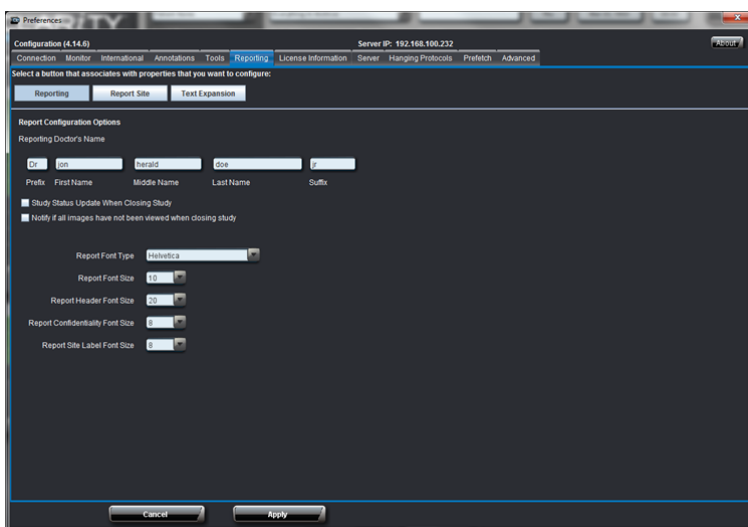

Reporting Doctors Name

In the **Report Writer** tab, the User may set the name of the physician that will appear in the report.



Note:

This is a per User setting. Each User can have his or her own name to appear in the report. Only Users with the user privileges of "doctor" can write reports. Please see the Clarity Configuration & Troubleshooting Manual for more information on creating new users and setting user privileges.

Study Status Update When Closing Study

When this option is checked, upon closing an image from the **Viewer**, a dialog box will appear asking the User to update the Study Status:

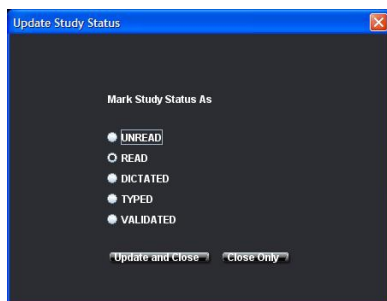


Figure 1: The Update Study Status dialog pops-up after closing a study in the Viewer window.

This option works in conjunction with the Filter Modifiers and Study Status workflow options present in the Worklist window.

Options

Update and Close: Updates the study status to the selected value, e.g. "Read", then close the study.

Close Only: Closes the study without changing the study status.

If a report is written with the option enabled, upon clicking the **Create** button, the report will be generated, the current patient will close, the study status will be set to **Validated** and the next patient in the worklist will be selected.

Notify if all studies have not been viewed

When this option is checked, if all images in the study have not been viewed, the Clarity Viewer will display a warning dialog notifying the User.

Report Font Type

The User may configure the body text font face used when writing reports in the Clarity Viewer. Select the desired font from the drop-down menu. Click the **Apply** button to save the changes.

Report Fonts Size

The User may configure the body text font size used when writing reports in the Clarity Viewer. Select the desired font size from the drop-down menu. Click the **Apply** button to save the changes.

Report Header Font Size

The User may configure the header text font size used when writing reports in the Clarity Viewer. Select the desired font size from the drop-down menu. Click the **Apply** button to save the changes.

Report Confidentiality Font Size

The User may configure the Report Confidentiality text font size used when writing reports in the Clarity Viewer. Select the desired font size from the drop-down menu. Click the **Apply** button to save the changes.

Report Site Label Font Size

The User may configure the Report Site Label text font size used when writing reports in the Clarity Viewer. Select the desired font size from the drop-down menu. Click the **Apply** button to save the changes.

Report Site

Configuration (Version: 4.14) Server IP: 192.168.100.234

Connection Monitor International Annotations Tools **Reporting** License Information Server Hanging Protocols Advanced


Select a button that associates with properties that you want to configure:

Reporting Report Site Text Expansion

Report Icon

Choose Icon Clear Icon

Current Icon



☐ Use referring physician's facility information

Site Name My Imaging Center

Site Address 222 West Main St
Tampa Bay, Florida 33323

Phone Number (665) 565-5656

Fax (645) 232-5454

URL http://mytestsiteimagingcenter.com

Confidentiality Agreement

The information contained in this communication is privileged and confidential information intended for the sole use of the medical facility named below. If the reader of this communication is not the intended recipient, you are hereby notified that any dissemination, distribution or copying of this communication is strictly prohibited and you are asked to notify the sending party immediately.

Report Body Elements

Number of Report Body Elements 6

Procedure Description

History

Findings

Impressions

Recommendations

Conclusions

Cancel Apply

Report Icon

Clicking the **Choose Icon** button will allow the User to select a JPEG, GIF, or PNG file of the practice or site logo to be placed in the report header. Clicking the **Choose Icon** button will bring up a file upload dialog from which the User may select the site logo. After selecting the desired graphic, click the **Save** button to save the icon. The **Clear Icon** button removes the current icon, allowing for the upload of a new icon.

Current Icon

Displays a preview of the current icon that is displayed in the Report header.

Site Name

Enter the Site or Practice name in the text field.

Site Address

Enter the Site or Practice address in the text field.

Phone Number

Enter the Site or Practice phone number in the text field.

Fax

Enter the Site or Practice fax number in the text field.

URL

Enter the Site or Practice website address in the text field. For example: <http://www.icrcompany.com>

Confidentiality Agreement


Enter the Site or Practice confidentiality agreement text in the text field.

Report Body Elements

The **Report Body Elements** allows the User to customize the elements in a Report. Each element can only appear once in a report (an **element** is Impressions, Findings, Procedure Description, Conclusions, etc.) Select the number of elements to appear in a report in the **Number of Report Body Elements** box, then select the heading for each element (For example, Impressions, Findings, Conclusions, etc).

Text Expansion

This feature facilitates the User to set a **Key Text** and its **Expansion Text**. The advantage of this feature is that the User just has to type the **Key Text** while writing a patient report and the application automatically pulls up its **Expansion Text**.

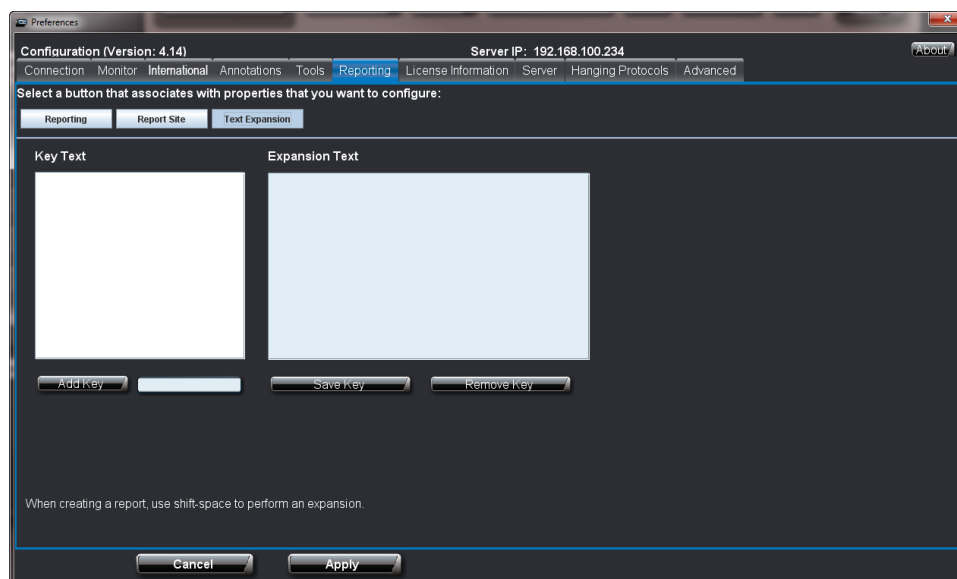

To use this feature, see the following steps:

1. Type a **Key Text** against the **Add Key** button and click **Add Key**.
2. The **Key Text** just added gets displayed under **Key Text**.
3. Select this **Key Text** and click in the box provided under **Expansion Text**
4. Type in the details for the **Key Text** and click **Save Key**.
5. Select a **Key Text** and click **Remove Key** to delete the **Key Text**.
6. Click **Apply** and restart the Clarity Viewer to make the changes effective.
7. To use this feature:
 1. Select a patient from the worklist and open any image.
 2. Click the **Reports** button on the left panel and the click **Write Report**.
 3. Type the **Key Text** wherever you want the **Expansion Text**.
 4. Press **Shift Key + Space Bar**.
 5. The **Key Text** will be converted into **Expansion Text**

License Information

This tab will display the software serial number, sales order number, number of modality inputs, number of LAN and Web viewers, and enabled Clarity modules. This information is especially important when contacting Technical Support.

Server

The server tab displays Clarity's server information.

Backup Errors

Everytime a user logs in, Clarity will check the backup status of the server. If an error occurs, Clarity will display a dialog box detailing the error information.

To turn off the backup error dialog, mark the **Do not show backup errors** check box. Similarly, uncheck the box to reactivate the warnings.




Note:

It is also possible to disable the messages in the dialog box.

Hanging Protocols

The Hanging Protocols tab allows the user to modify anatomy window/level presets and manage saved protocols.

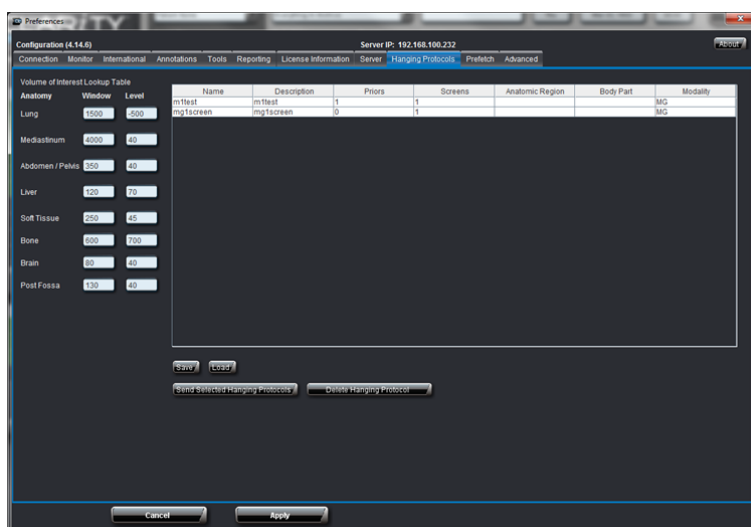

On the left side of the window, the user can set the presets for the window/level settings for each anatomy. The specific value can be modified in each of the anatomy's fields.

After logging in, the user can see his or her saved hanging protocols on the right side of the window. The table displays information including the name, description, and modality for each hanging protocol.

Clicking the **Save** button will save the selected hanging protocol in DICOM format, while clicking the **Load** button will load a saved hanging protocol. Clicking the **Send...** button will transfer a selected hanging protocol to another server. In this manner, a user may share hanging protocols with another user.

Click the **Delete...** button to remove a hanging protocol.

Prefetch


Prefetch allows Clarity to automatically download studies to a workstation based on rules that the user creates. Because these studies have been downloaded, they will open much faster on the workstation.

Accessing Prefetch Rules:


1. Open the configuration in the Clarity Viewer.
2. Click on the Prefetch tab.

Creating a Prefetch Rule:

1. Click the “Add” button underneath the prefetch rules table.

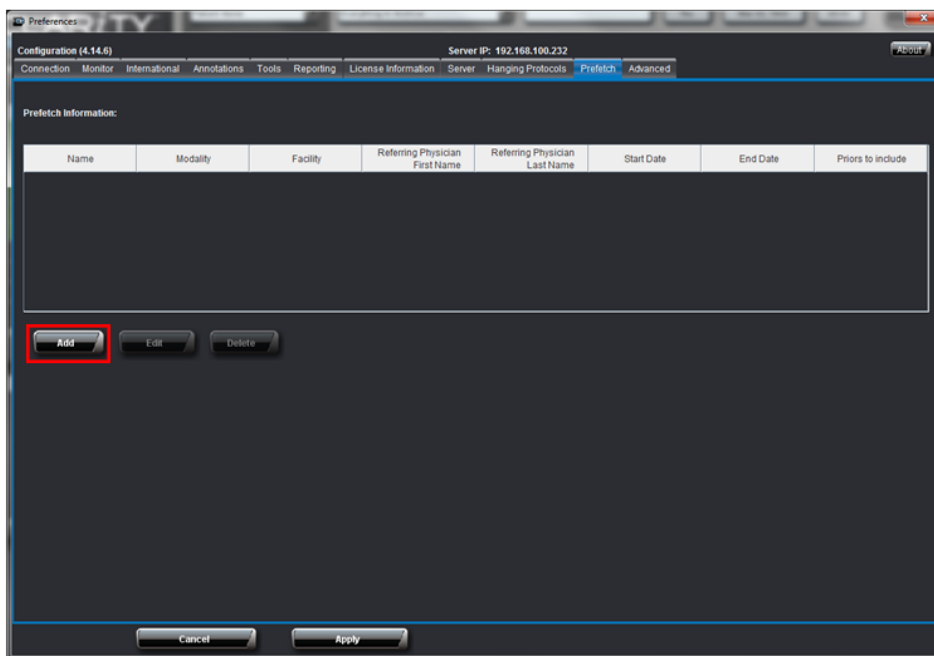

2. In the new window that pops up, type in a name for the rule and select the desired settings [note with the finger icon for following two sections]:

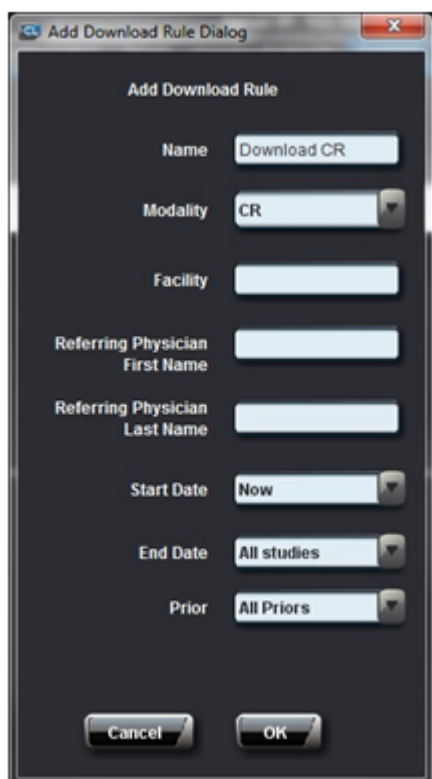

- a. If a name is not chosen for the rule, you will not be able to save it.
- b. Example, to set up a rule that will cache CR studies created in the last 24 hours:

- name: Download CR
- modality: CR
- start date: One day ago
- end date: Now


3. Click the “Add” button at the bottom of the rule information box to create the rule.

After Prefetch rule has been created it will start fetching studies from Clarity PACS server based on rule criteria, images will be fetched to C:\iCRco\images folder location, unless user has any predefined configuration for image cache.

Clarity PACS will have look for studies every 5 minutes based upon rule criteria. If any studies are available it fetches those studies from Clarity PACS Server.

Editing a Prefetch Rule:

1. Click the desired prefetch rule in the table of available rules.
2. Click the “Edit” button underneath the table.

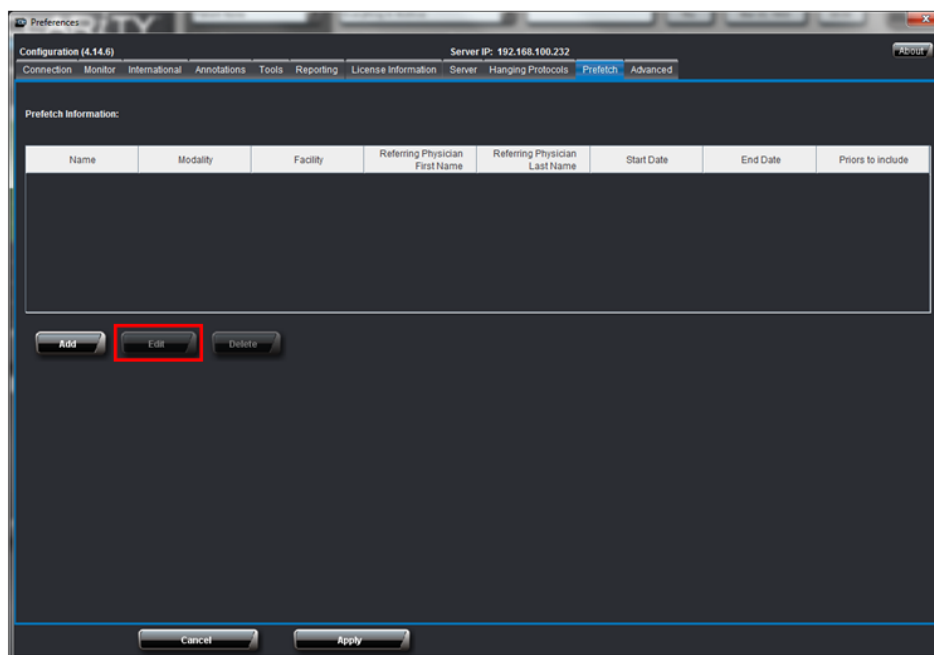


3. Enter/modify the rule settings and click the “Add” button to save the changes to the rule after rule have been edited, Clarity PACS will look for studies which can be fetched if any images pass the rule criteria those images will be fetched.

Deleting a Prefetch Rule:

1. Click the desired prefetch rule in the table of available rules.
2. Click the “Delete” button underneath the table to remove the prefetch rules.

Advanced

The Advanced tab allows the users to select the desired representation for person names, auto-advanced worklist, viewer logger level, image cache drive and percent confidence in memory prediction.

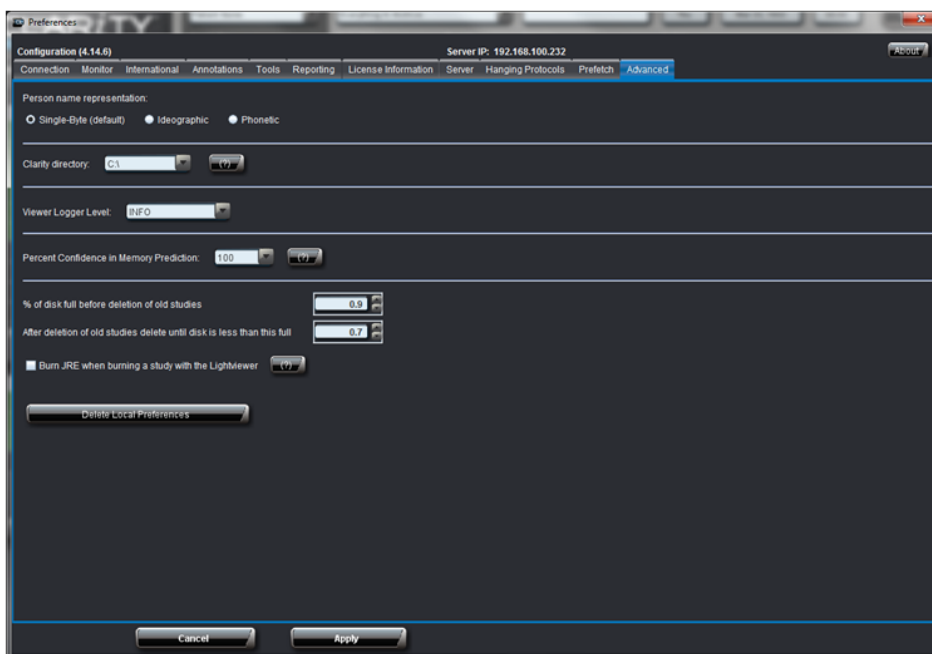

Desired Representation for Person Names

The person names can be written in three different characters. The user can select any representation out of **Single-Byte (which is default)**, **Ideographic**, and **Phonetic** to get the desired display of the person name.

Clarity Directory

The Clarity Directory setting allows the user to determine the location of the Clarity root directory. Changing the directory will change where the cache files are saved. After switching to a new directory, restart Clarity.



Important:

Changing to a new directory will delete the cached files in the old directory.



Note:

The default directory is C:\. If the permission settings on the computer do not allow data to be written to the C Drive, then Clarity will use the APPDATA location. Clarity directories written to APPDATA can be found under C:\Documents and Settings\[username]\Application Data\ICRCO on Windows XP and C:\Users\[username]\AppData\Roaming\ICRCO on Windows 7. The APPDATA folder is hidden by default on Windows.



Note:

The pre-fetch feature will only work with the C Drive.



Note:

Clarity Light Viewer will always use APPDATA and the directory cannot be changed.

Viewer Logger Level

The User can select the Viewer Logger Level as **Debug**, **Info**, **Warn**, **Error**, and **Fatal**. By default, **Info** is selected and it means that the log report for Clarity viewer will include all the logger levels provided below the Info level in the combo box. So, the log report shows all the levels below the selected level in the combo box including the selected level itself in the log report. So, if the user selects Debug level, then all the levels will be shown in the log report.


Percent Confidence in Memory Prediction

This option is set to 100 by default. To get more information on what it actually is, click on the button having (?) on it. An **Information Message** box pops out giving the details about percent confidence in memory prediction.



Important:

It is highly recommended that this value remain at 100 percent. If necessary, the value should only be modified by qualified personnel.

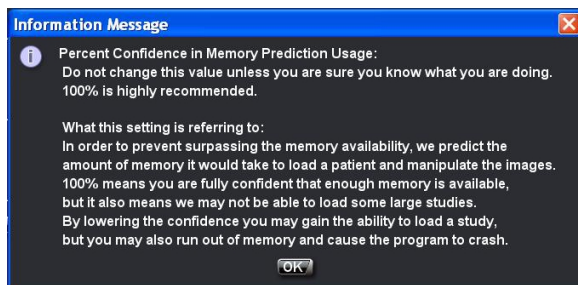


Percent of Disk Full before Deletion

Clarity automatically caches opened studies, eventually filling up the cache. In order to prevent the cache from filling to capacity, the program will automatically trim older images.

The user can modify when Clarity will begin deleting older images and how much space it will try to free up.

It is possible to delete local preferences stored in the Windows registry such as monitor layout and user preferences.

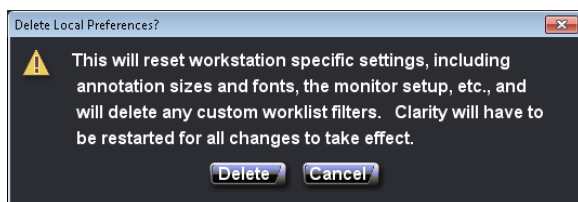

Burn JRE when Burning a Study

When this option is selected, Clarity will include the Java Runtime Environment (JRE) software with the Lightviewer when burning a study to disk. This is only necessary if the exported study will be opened on a computer lacking an internet connection.



Note:

This will result in a significantly longer burn time.

The setting is off by default.

About Button

The About This Software dialog can be accessed by clicking on the **Configuration** button. This will bring up the **Preferences** dialog. From there, click the **About** button on the top, right corner. The About button will pop-up the End User License Agreement for the Clarity Viewer application.

Archive Storage Limit

Available Space

The amount of storage space available for the PACS Archive is controlled by a value stored on the security dongle. This storage size will be based on the available hardware when the PACS system was purchased, or set to a limit as defined by the sale. For Clarity PACS version 4.12 and earlier, the archive storage limit will remain unchanged.

You can see what your current Archive storage limit is by visiting the **License Information** tab in the **Preferences** dialog. Typically, the Archive storage limit is 1, 2, 4, 6, 8, or 12 TB.

Storage Indicator

Archive Storage at 1.21%

The colored **Archive Storage** indicator at the top of the main window displays the amount of available Archive storage space.


This indicator shows, as a percentage, the current size of the archive against the archive size limit using the following formula:

$(\text{current archive size}) / (\text{archive size limit}) * 100.0 = \text{Percentage of the Archive that is Full}$

The indicator will change color depending on how full the archive is:

- Green - the archive is less than 90% full
- Yellow - the archive is between 90% and 93% full
- Orange - the archive is between 93% and 95% full
- Red - the archive is greater than 95% full

Warning Dialog

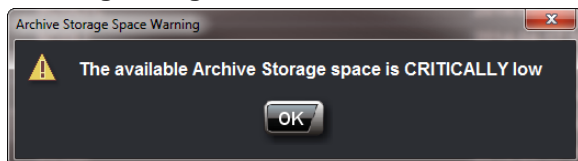

A warning dialog will appear during login if the archive is greater than 93% full (Orange or Red state). If the archive reaches 100% capacity, users may still be able to view images, but cannot store new studies on the archive.

If the archive reaches 100% capacity, it is possible to free space within the archive. Alternatively, you may contact iCRco for additional options.

Contact iCRco

You can contact iCRco by clicking the colored **Archive Storage** indicator, which will open your default web browser to the iCRco Clarity Support Inquiry page. The page will automatically display the archive's storage state and the security dongle's serial number. Fill in the required fields and submit the information. iCRco will contact you regarding your archive.

Warning Lights

Clarity will display warning lights when the warranty is expiring, when the dongle license is expiring, and when the disk storage is approaching maximum capacity.

It is possible to hide the warning lights by clicking the "X" to the right of the warnings. The lights will appear each time Clarity starts.

Expired Warranty

Your warranty has expired

When the user's warranty is about to expire, Clarity will notify the user the number of days until expiration. The warning message is displayed at the top, left corner of the screen. To renew the warranty, call (310) 921-9559.



Note:

The warning message color will vary depending on the time until expiration.

- Yellow - Will expire within 45 Days
- Orange - Will expire within 30 Days
- Red - Will expire within 14 Days and after expiration

License Expiration

Your license will expire in 6 days

The Clarity Viewer uses a dongle licensing verification system. When the user's license is about to expire, Clarity will notify the user the number of days until expiration. The warning message is displayed at the top of the screen. To update the dongle license, the user should contact the dealer or iCRco by calling (310) 921-9559.



Note:


The warning message color will vary depending on the time until expiration.

- Yellow - Will expire within 45 Days
- Orange - Will expire within 30 Days
- Red - Will expire within 14 Days and after expiration

Logging Out & Quitting

To log out of the Clarity Viewer, click the **Log Out**  button, located in the upper, right-hand corner of the Worklist window. Logging out of the Viewer leaves the application open, but will require the User to enter a name and password to be able to access data again.

To quit the Clarity Viewer, click the **Quit**  button, located in the lower, right-hand corner of the Worklist window. Quitting the Viewer means the User will have to re-open the application to access data again.

To minimize the Clarity Viewer, click the **Minimize**  button, located in the lower, right-hand corner of the Worklist window. Minimizing the Viewer allows the user to hide the Viewer without exiting the program. This can be useful if the user needs to utilize other programs and doesn't want to quit out of Clarity.

Chapter

3

Operation and Usage


Workflow Controls




Note:

Workflow control buttons may appear grayed out a certain times. A grayed out Workflow button means that the User may not use the button's function at the present time (e.g., when exporting images, the Clarity Viewer can not be quit, thus the quit button is grayed out).

Worklist


The Worklist button  returns the User to the Worklist window.

Viewer


The Viewer button  returns the User to the Image Viewing window.

Send/Save Study


From the Worklist:

Clicking the **Send Study** button  with patient{s} selected in the Worklist will bring up the DICOM Send dialog. The user will need to click "Yes" when the program opens a dialog box asking if the user wants to continue. From the DICOM Send dialog, the User may DICOM send the studies to a different target, and configure new targets.

From the Image Viewing Window:

Clicking the **Save Study** button  while in the Viewer allows the User to save image manipulation settings (sans Pan and Zoom settings) and annotations. The User may save multiple different sets of annotations on the same image.

Export

The Export button  opens a dialog for exporting the selected studies. Studies can be exported to a directory, CD/DVD, or network drive. For more information, see Section [Exporting Studies](#) on page 70.

Print

The Print button  opens the current image(s) with the DICOM print application.


Quit

The Quit button  exits & closes the Clarity Viewer.

Minimize

The Minimize button  hides the Clarity window without exiting the viewer.

Search

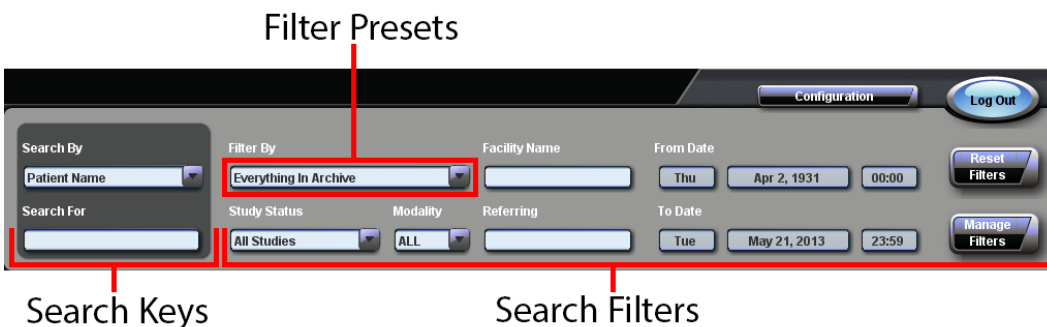

The screenshot shows the Search interface with three main sections labeled with red lines and text:

- Filter Presets:** A red line points to the 'Filter By' dropdown menu, which currently shows 'Everything In Archive'.
- Search Keys:** A red line points to the 'Search By' dropdown menu (showing 'Patient Name') and the 'Search For' text input field.
- Search Filters:** A red line points to the 'Study Status' dropdown (showing 'All Studies'), the 'Modality' dropdown (showing 'ALL'), and the 'Referring' text input field.

Other visible elements include 'Facility Name', 'From Date' (Thu Apr 2, 1931 00:00), 'To Date' (Tue May 21, 2013 23:59), 'Configuration', 'Log Out', 'Reset Filters', and 'Manage Filters' buttons.

Search Keys

The Search Keys, located in the upper, left-hand corner, allow the User to search by patient last name, patient ID, accession number, or study ID. Select one of these from the **Search By** drop-down menu, and then enter the search criteria in the **Search For** box below. As soon as the User begins typing, the results of the search will appear in the **Patient / Study Search Results** Worklist, located near the middle of the screen.

Search Filters


Search Filters are used in conjunction with Search Keys to help narrow the results of a search. The Search Filters can filter by study status, modality, facility name, date, and/or referring physician.

Filter Presets

Filter Presets are user-defined, allowing the User to define an optimal workflow. The Filter Presets can be accessed in a drop-down menu in the upper left part of the Search Filters box. Clicking the drop-down menu will reveal a list of the default Search Filters.

Creating and Editing Filter Presets

The User can edit & define new Filter Presets by clicking on the **Manage Filters** button.

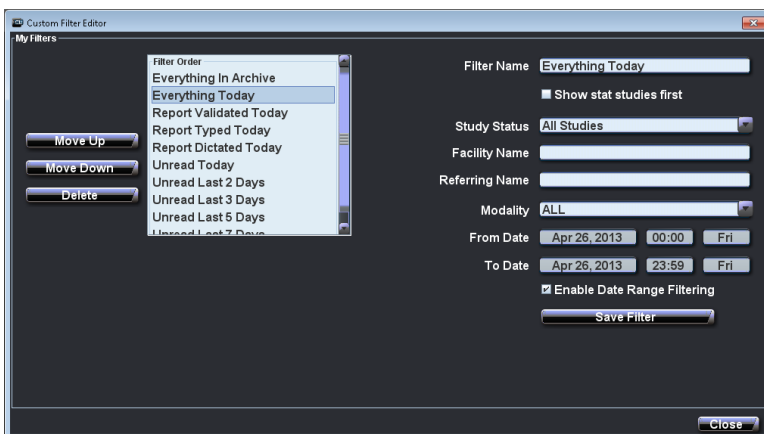

The screenshot shows the 'Custom Filter Editor' dialog box. It has a 'My Filters' list on the left with buttons 'Move Up', 'Move Down', and 'Delete'. The list contains: 'Filter Order', 'Everything In Archive', 'Everything Today', 'Report Validated Today', 'Report Typed Today', 'Report Dictated Today', 'Unread Today', 'Unread Last 2 Days', 'Unread Last 3 Days', 'Unread Last 5 Days', and 'Unread Last 7 Days'. On the right, the 'Filter Name' is 'Everything Today', and there is a checkbox for 'Show stat studies first'. Below that are fields for 'Study Status' (All Studies), 'Facility Name', and 'Referring Name'. Then 'Modality' (ALL), 'From Date' (Apr 26, 2013 00:00 Fri), and 'To Date' (Apr 26, 2013 23:59 Fri). There is a checkbox for 'Enable Date Range Filtering' and a 'Save Filter' button. A 'Close' button is at the bottom right.

**Note:**

The default filters cannot be modified.

New Filter Preset:

1. From the **Worklist** Window, click **Manage Filters**.
2. Enter the desired filter name in the **Filter Name** text box. Make sure the new name is not already entered as a filter name.
3. Enter the desired filter criteria. The User does not have to fill in all filter criteria, e.g., if the User desired a filter for all CR studies, then select the modality from the **Modality** drop-down menu, and select the date ranges.
4. Click the **Save Filter** button.

Editing a Filter Preset:

1. From the **Worklist** Window, click **Manage Filters**.
2. Highlight the filter to edit in the **Filter Order** box.

**Note:**

Default filters cannot be edited.

3. Edit the desired Filter criteria.
4. Click the **Save Filter** button.

Show Stat Studies First

When the user checks this option, the filter will automatically reorder the studies so that studies marked as "stat" are shown at the top of the list.

**Note:**

It is possible to mark a study as "stat" by right-clicking on the study in the worklist and selecting **Change Stat**. To unmark a study, simply select **Change Stat** on the study once more.

Status	Modalities	Series	Images	Patient Name
	PRICR	2	30	25, 5555555
	PRIDX	3	5	นาง SIM
	PRICR	2	111	1.7.0.6, TEST
	PRICR	2	5	EE, EEEE
	PRICR	2	48	VET, VET
	PRICR	2	7	PLATA, ANTONIO
	PRIDX	2	325	นาง, กิ่งหมอก
	PRICRISR	8	31	ASF, ASD
	PRICR	7	54	ASDFF, ASF
	PRICR	10	108	ASDFG, ZCXZC
	PRICR	10	18	ASFAF, FASFF
	PRIDXISR	4	5	อวิษฐ์, อธิชาญนพงศ์

Enable Date Range Filtering

This option directs the filter to only display studies within a specified date range.

Worklist

Filtered Studies

Status	Stat	Study Status	Patient ID	Patient Name	Date of Birth	Sex	Study Description	Study ID	Modalities	Study Date	Series	Images	Facility
No	Unread	10056	ORIENTATION TEST	2013-01-01	F	CHEST, LSPNIE, C SPNIE		DX	2013-05-20 15:55:57.0	3	3	ICRCO	
No	Unread	10227	CHANNEL TEST 32 BIT	2013-01-01	F	BREAST		MG	2013-04-12 07:59:18.0	17	17	ICRCO	
No	Unread	10225	CHANNELS TEST 64 BIT	2013-01-01	F	BREAST		MG	2013-04-11 07:59:51.0	17	17	ICRCO	
No	Read	490060740	CHL. FUSION	1928-10-02	M	CHEST R	560045624	PRIOX	2013-04-10 09:03:16.0	2	5		
No	Read	490033077	CHL. DYNAMICWAVE	1955-10-29	M	ABDOMEN	503921	PRIOX	2013-04-04 16:44:11.0	3	6		
No	Unread	490011212	CHL. WOLFFER	1934-01-01	F	CHEST	524503	DX	2013-03-27 16:34:49.0	1	1		
No	Unread	490011212	CHL. WOLFFER	1934-01-01	F	CHEST R	524503	DX	2013-03-26 10:54:18.0	1	1		
No	Unread	10184	NAMC CHEST	1952-06-05	F	CHEST, KNEE		PRIOX	2013-03-11 11:46:28.0	3	18	DMCH	
No	Unread	BENJAMIN2287	BENNETT, JOSEPHSON	1934-12-23	M	CT CHEST W/O	12601	CT	2012-12-19 09:54:06.0	6	263	THE MEDICAL MALL	
No	Unread	10001	TEST, DCMIND	2012-01-01	F	ABDOMEN		CR	2012-11-30 09:07:38.0	1	1	ICRCO	
No	Unread	23423	VOGEL DOG	2012-01-01	F	TIBI/FIB L		PRIOX	2012-11-15 17:45:09.0	3	5	ICRCO	
No	Unread	10007	VISUAL MEDICA	2012-01-01	F	BREAST		MG	2012-10-03 09:46:14.0	5	5	D	

Patient Studies

Status	Stat	Study Status	Patient ID	Patient Name	Date of Birth	Sex	Study Description	Study ID	Modalities	Study Date	Series	Images	Facility	Patient PK	Study PK	Accession
No	Unread	10184	NAMC CHEST	1952-06-05	F	CHEST, KNEE		PRIOX	2013-03-11 11:46:28.0	3	18	DMCH	305	450		

Page: (1) Studies 1 to 300

Navigation buttons: Work List, Viewer, Send Study, Export, Print, Quit

Filtered Studies

Filtered Studies appear below the Search Keys and Filters. It's possible to sort the table by clicking any of the column headers.

Patient Studies



Note:

The Patient Studies Worklist will only appear with screen resolutions of 1280 x 1024 or above.

Patient Studies appear below the Filtered Studies. It includes all studies related to the patient selected on the Filtered Studies table.

Thumbnails

The thumbnail previews are shown at the bottom of the screen when a study is selected in either the Filtered Studies or Patient Studies Worklist.

Status

The Status column uses colors to indicate the current loading state of the study.

- Black - The image is located on the server.
- Blue - The image is queued to be downloaded from the server.
- Yellow - The image is currently downloading from the server.
- Green - The image has been retrieved from the server and is stored in the cache. It can be viewed.



Note:

It is possible to query and retrieve multiple studies without opening them.

Viewing Selected Studies

From the Worklist:

To view a selected study, the User may double click the desired study in the Sorted or Patient Studies, or the User may double click the preview thumbnail at the bottom of the Worklist.

From the Image Viewer window:

To view a selected study, mouse over the bottom part of the window to reveal the thumbnails associated with the study, then drag and drop the desired image into the image manipulation window and the image will load.

Lossless and Lossy Images

Clarity uses lossless algorithms for image compression. The software supports JPEG-LS Lossless (JLSL) and JPEG Lossless (JPLL) compression services. Lossless compressed images can be used for primary image interpretation.

It is possible for Clarity to receive images from outside sources that were compressed using lossy compression services. The software can accept JPEG Lossy (JPLY) images from other sources, but does not apply this compression service. Lossy images may not be used for primary image interpretation.

The user may determine if a lossy compression service was used on an image by checking the DICOM tags. To check:

1. Open the image in the Viewer.
2. Press **Ctrl + D** on the keyboard. This will open a dialog box with the list of the image's associated DICOM tags.
3. Locate the DICOM tag labeled **(0028,2110)**. This is the Lossy Image Compression tag.



Note:

If this tag does not exist, the image was not subjected to lossy compression.

4. If the tag exists, examine the tag's value. If the value is 00, then the image was not subjected to lossy compression. If the value is 01, then the image was subjected to lossy compression and may not be used for primary image interpretation.

Tag	Description	Value
(0028,0103)	Pixel Representation	0
(0028,0106)	Smallest Image Pixel Value	17292
(0028,0107)	Largest Image Pixel Value	65535
(0028,0301)	Burned In Annotation	NO
(0028,1040)	Pixel Intensity Relationship	LOG
(0028,1041)	Pixel Intensity Relationship Sign	-1
(0028,1050)	Window Center	45219
(0028,1051)	Window Width	16494
(0028,1052)	Rescale Intercept	0
(0028,1053)	Rescale Slope	1
(0028,1054)	Rescale Type	US
(0028,1056)	VOI LUT Function	LINEAR
(0028,2110)	Lossy Image Compression	01
(0038,4000)	Visit Comments	
(0040,0244)	Performed Procedure Step Start Date	20131218
(0040,0245)	Performed Procedure Step Start Time	133643
(0040,0253)	Performed Procedure Step ID	
(0040,0254)	Performed Procedure Step Description	Shoulder
(0040,0275)	Request Attributes Sequence	
(0040,0007)	Scheduled Procedure Step Description	
(0040,0009)	Scheduled Procedure Step ID	
(0040,1001)	Requested Procedure ID	

Image Layout and Navigation

Moving the mouse to the left hand side of the screen will make the tool bar appear so that the User can select the necessary image manipulation tool.

Moving the mouse to the bottom of the screen will make the thumbnails associated with the study appear. To load a different image into the image manipulation window, drag the thumbnail into the image manipulation window and it will automatically load.

Locking the Tool bar



The User has the option to make the tool bar always visible. Clicking the **Lock** button at the bottom of the tool bar will cause the tool bar to be always visible. Deselecting the button will return the tool bar to its hidden state.

Default Mouse Actions

When the Viewer window is first opened, the mouse buttons are automatically mapped to image manipulation tools. The mapped mouse button is indicated by a mouse icon on the upper, right-hand corner of the tool's icon (illustrated in the graphic below). The default mapped mouse buttons can be changed. See Section [Select Default Tools Per Modality](#) on page 28 for more information.

Holding down the right mouse button and moving it vertically on the screen will adjust the brightness of the image. Holding down the right mouse button and moving it horizontally on the screen will adjust the contrast of the image.

Holding down the left mouse button and moving the mouse in any direction will allow the User to pan around the image.



If the User has a scroll wheel mouse, moving the scroll wheel up will zoom into the image, while moving the scroll wheel down will zoom out.


Each mouse button is re-mappable to any tool that has an interactive function. To re-map a mouse button, move the mouse over the left side of the screen to make the tool bar pop-up. Locate the tool the User wishes to map to the

mouse button, move the mouse over that tool, then click the tool with the mouse button the User wishes to map. For example, if the User wishes to map the Mag tool to the right mouse button, move the mouse over the Mag tool button, then right mouse click on the Mag tool button. You will see a small mouse icon appear in the upper right-hand corner of the Mag tool button, this means the User has successfully re-mapped the mouse button. To use the newly mapped Mag button, move the mouse over the image at the point the User wishes to magnify, then right click.

Default Image Layouts

The User may specify, on a per modality basis, the default layout of images in a study. Please see Section [Default Image Layouts](#) on page 47 to set this option. If enabled, this preference will override those specified below.

Study Layout

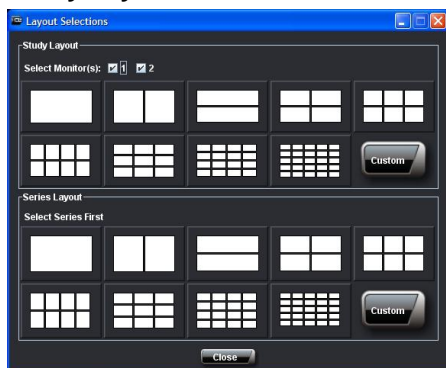

The Study Layout is used to view multiple images on one screen. To use the Study Layouts, first select which monitor to apply the layout selections to (the monitor is select when there is a dashed, white border just inside the image viewing pane). Then select the layout view to use, and the image manipulation window will split into the desired number of panes. Then drag the images from the thumbnail box into the viewing panes. To return to viewing one image, select the monitor to apply the layout to, then click the single box in the upper left hand corner of the **Layout Selections** dialog box.

Series Layout

The Series Layout is used to view multiple images in a series, such as images from the CT or MR modality. To use the Series Layouts, first select the monitor to apply the layout to, then select the layout view to use, and the image manipulation window will split into the desired number of panes. The images in the series will then populate the pane. To return to viewing one image, click the single box in the upper, left-hand corner of the layout selections dialog box. Images of the CT modality will automatically open four viewing panes, and the MR modality will open nine viewing panes. The User may change the number of panes for the CT and MR modality by clicking a different layout view from the Image Layouts dialog.

Next & Previous Image



These buttons move to the next and previous images in the patient study when inside an image manipulating window. The buttons can be revealed by hovering the mouse over the upper, right-hand corner or the image manipulation window. The left and right arrow keys will also advance the User through the images in a series. Hotkey: Right Arrow: Next Image, Left Arrow: Previous Image.

Next & Previous Series

These buttons move the User to the next and previous series in a study. The buttons can be revealed by hovering the mouse over the upper, right-hand corner or the image manipulation window. The up and down arrow keys will also advance the User through the series in a study.



Note:

You can move to the Next Series by pressing the Up Arrow and move to the Previous Series by pressing the Down Arrow.

Image Manipulation Functions

Pan



The Pan tool allows the User to move the image around the screen. This is especially useful if the User is zoomed into an image and needs to view another section of the image. By default, the Pan tool is mapped to the left mouse button. To use the Pan tool, first select it from the tool bar. Then move the mouse over the image. Click and hold down the mouse button, then move the mouse in the desired direction to pan. Hotkey: P

Zoom



The Zoom tool is used to increase or decrease the magnification of the image. By default, it is mapped to the middle mouse button. To use the Zoom tool, first select it from the tool bar, then mouse over the image. Click, hold, and move the mouse vertically to change the zoom level. Hotkey: Z

Mag



The Mag tool allows the User to temporarily magnify part of the image. To magnify a part of the image, first select the Mag tool from the tool bar. Mouse over the image, then click and hold the mouse button. A box will appear with the magnified section of the image. While holding down the mouse button, the User can use the up and down arrow keys to increase or decrease the magnification inside the box, and the left and right arrow keys to increase the size of the magnification box. Hotkey: G

Window/Level



The Window/Level tool allows the User to adjust the brightness and contrast of the image. By default, this is mapped to the right mouse button. To use the W/L tool, first select it from the tool bar. Then hold down the mouse button

and move the mouse horizontally to adjust the contrast of the image and move the mouse vertically to adjust the brightness of the image. Hotkey: W

Smart Window/Level



The Smart Window/Level tool allows the User to do an automatic Window/Level optimization determined by a user-selectable box. To use the Smart W/L, select the tool from the tool bar, then click, hold, and drag to draw the box. Release the mouse button when the box is a sufficient size. The box will optimize the W/L setting for the selected area.

Invert



The Invert tool will invert the colors of the image. To use the Invert tool, first locate it in the tool bar. Then single mouse click to invert the image.

Reset



The Reset button will return the image to the state it was in when it was opened. It will remove any Window/Level, Zoom, Invert, and Rotation manipulations done to the image. It will not, however, remove annotations on the image. To use the Reset tool, locate it in the tool bar, then click it once to reset the image.

Image Layouts




The image layout tool allows the User to compare two or more images side by side. Clicking on the Image Layout button will pop-up the **Layout Selections** dialog box.

Predetermine Window/Level



The Predetermined Window/Level tool applies a specified Window/Level values to the image that had been previously been set by the user.

Mark Key Image

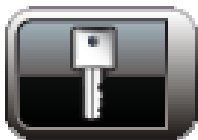

The Mark Key Image tool classifies the current viewed image as important. All marked images are attached to reports.

Key Image Montage



The Key Image Montage tool changes the display to only show the marked images.

Cine Tools

Scroll



The Scroll tool allows the User to scroll through a series of images. To use the scroll tool, first select it from the tool bar. Then mouse over the image and move the mouse vertically. Hotkey: S

X-Ref



The X-Ref tool allows the User to show cross-reference lines on two overlapping, anatomic views from the same study. To use the X-Ref tool, first select it in the tool bar. Then click on the image that the User wants cross-referenced, and the cross-reference lines will appear. Hotkey: X

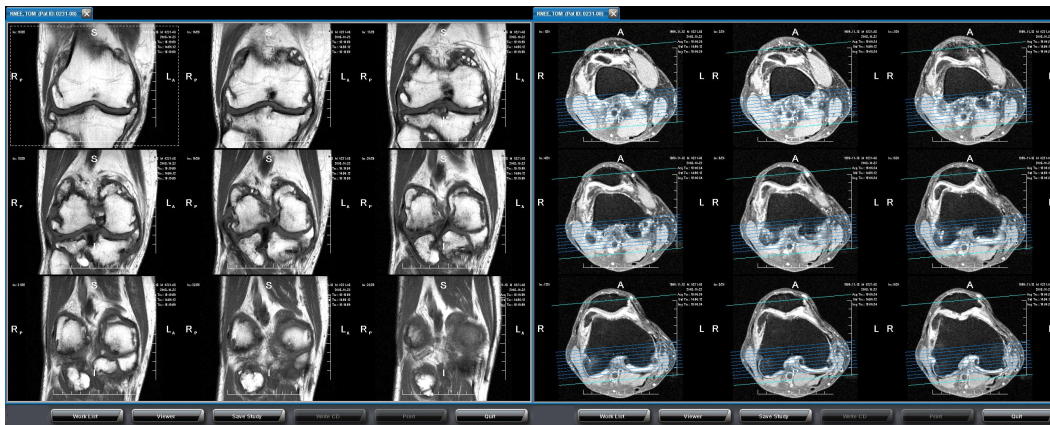


Figure 2: Cross-references lines in the Clarity Viewer

Linking



The Linking tool allows the User to apply the cine operation to two different series that were acquired using the same view. To use the Linking tool, first select it from the tool bar. Then click on the on one frame of the study that is to be linked. Use the cine play-back tools to play both series at the same time. Hotkey: L

Loop Forward



The Loop Forward tool will play back, in a cinematic style, a series with multiple images from the first image to the last image and continue to play them until it is halted.

Loop Backward




The Loop Backward tool will play back, in a cinematic style, a series with multiple images from the last image to the first image and continue to play them until it is halted.

Cine Forward & Back



The Cine Forward & Back tool will play back, in a cinematic style, a series with multiple images from the first through the last image, then from the last image to the first and continue to play them until it is halted.

Pause/Play

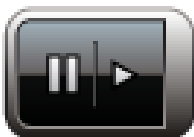

The Pause/Play tool will pause the play back from the Loop Forward, Backward, and Forward & Back tools.

Frames Per Second Selection



The Frames Per Second (FPS) slider allows the User to adjust the play back rate for the Loop Forward, Loop Backward, and Forward & Back tools.

Rotate Tools

Flip



The Flip tool will vertically invert the image. To use the Flip tool, locate it in the tool bar, then left mouse click once to flip the image.

Mirror



The Mirror tool will horizontally invert the image. To use the Mirror tool, first locate it in the tool bar, then left mouse click once to mirror the image.

Rotate Left & Right



The Rotate Left and Right tools will rotate the image in 45 degree increments to the left or right depending on which button the User chooses.

Zoom to Full



The Zoom to Full tool magnifies the image to a level in which the entire image can be seen.

Zoom to Pixels



The Zoom to Pixels tool magnifies the image to a level in which one pixel in the image is equivalent to one pixel on the screen.

Measurements

Drawing Over Existing Annotations

Once an annotation has been created, clicking it will allow you to move or resize it. In order to draw a new annotation right on top of an existing annotation, press and hold the "shift" key. Any existing annotations will dim and you will be allowed to draw right on top of existing annotations as long as the "shift" key is depressed. The amount of dimming can be controlled from **Configuration > Annotations > Shift Key Opacity**.

key. Any existing annotations will

Measurement



The Measurement tool allows the User to make a linear measurement directly on the image. To use the Measurement tool, first select the tool from the tool bar, then mouse click and hold at the point where the measurement starts, then while holding the mouse button, drag the mouse to the end point of the measurement and release the mouse button. While dragging, the User will see the measurement. You can move the measurement around the image by selecting it (the User will know the measurement is selected when four dots show up around the measurement), then dragging it around the screen. To remove the measurement, select it and hit the Backspace key. Hotkey: M

HU



The HU tool, or Hounsfield Unit, will convert the gray scale value on the image to a pixel value/Hounsfield Unit. To use the HU tool, first select the tool from the tool bar, then move the mouse to the desired area and click. A cross will appear, displaying the Hounsfield Unit at that point. Hotkey: H

HU ROI



The HU ROI tool, or Hounsfield Unit Region of Interest, will take a selected region, average the gray scale units for the region, then convert the average gray scale unit into a Hounsfield Unit. The tool will display the Hounsfield Unit measurement, the standard deviation of the pixel value, and the area of the circle in cm^2 . To use the HU ROI tool, first select it from the tool bar. Then click, hold, and drag out the box. Release the mouse button when the HU ROI box is the desired size. Hotkey: R

Angle



The Angle tool allows the User to measure angles on an object in the image. The Angle tool is able to display the interior or exterior measurement of the angle.

To use the Angle tool, first select it from the tool bar, then move your mouse to where the vertex of the angle will be, mouse click, hold, and drag out one leg of the angle. When the length of the leg is satisfactory, release the mouse button. The second leg of the angle is drawn by moving the mouse to the desired end point of the second leg and mouse clicking. This will set the angle.

If the User wishes to display the interior angle measurement, move the mouse to the acute side of the angle and mouse click, or if the exterior measurement is necessary, move the mouse to the obtuse side of the angle and mouse click.

45° Angle Snap Feature

The 45° Angle Snap feature allows the user to precisely measure angle from the horizontal or vertical line by incrementing the angle every 45° between the first leg of the angle to the second leg of the angle.

To activate the angle snap feature, depress the Ctrl key as the second leg of the angle is drawn by moving the mouse to the desired end point of the second leg.

Complementary Angle

If the user wishes to display the complementary angle measurement, click on the vertex that displays the angle and move it around to display the interior angle, the exterior angle, and the complementary angles.

Cobb Angle



The Cobb Angle tool allows the User to draw a Cobb Angle on the image. To use the Cobb Angle tool, first select it from the tool bar, then mouse-click, hold, and drag to draw the first line. Release the mouse button when the first line is sufficiently long. Then mouse click, hold, and drag to draw the second line. Release the mouse button when the second line is sufficient in length. To set the interior or exterior measurement, mouse over the acute or obtuse side of the Cobb Angle, and mouse click again. The measurement will appear.

Center Line



The Center Line tool allows the User to annotate, in four different methods, a center line or point on the image. The User may select the Center Line annotation type by navigating to the **Worklist > Configuration > Tools** tab. To use the Center Line tool, first select it from the tool bar, then:

Two Points no Line (click and drag)

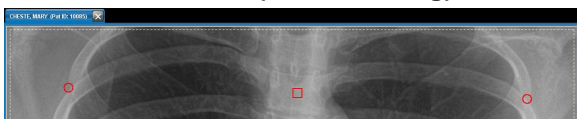


Figure 3: Center Line Tool: Two Points Without a Line

To use the Center Line tool with two points and no line, mouse click at the desired start point, hold, and drag the Center Line annotation to the desired length. When the Center Line is correctly positioned, release the mouse button. The square indicates the center point between the two circles.

Two Points with Line (click and drag)

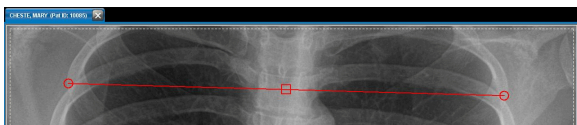


Figure 4: Center Line Tool: Two Points With a Line

To use the Center Line tool with a line, mouse click at the desired start point, hold, and drag the Center Line annotation to the desired length. When the Center Line is correctly positioned, release the mouse button. The square indicates the center point between the two circles with a straight line passing through all points.

Four Points

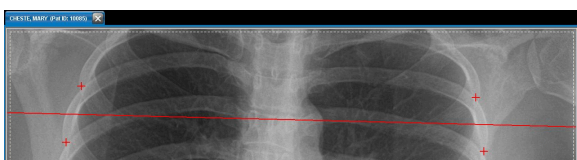


Figure 5: Center Line Tool: Four Points With a Line

To use the Center Line tool with four points, mouse click at each of the desired points. The four point version of the Center Line finds the center point between the first set of two clicked points, and the center point between the second set of two points, then draws a line through the two center points.

Eight Points with Angle

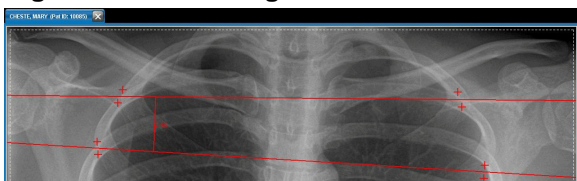


Figure 6: Center Line Tool: Eight Points With an Angle

To use the Center Line tool with eight points and an angle, mouse click at the first four desired points. The Center Line finds the center point between the first set of two points, and the center point between the second set of two points, then draws a line through the two center points. Repeat the process for the second set of four points and the angle of the two intersecting lines will be shown.

Heart to Lung Ratio



The Heart to Lung Ratio tool allows the User to find the ratio of the heart to lung size. To use the Heart to Lung Ratio tool, first select the it from the tool bar, then use the mouse to click, hold, and drag the first measurement, then release the mouse. Use the mouse again to mouse click, hold and drag for the second measurement. Release the mouse button when the length is satisfactory. The ratio of the two measurements will be displayed automatically.

Annotations

Toggle Annotations



The Toggle Annotations tool allows the User to hide and show annotations & measurements. To hide annotations, locate it in the tool bar, then click the **Toggle Annotations** button. Note that new annotations cannot be drawn while the annotations are hidden. To show the annotations, click the **Toggle Annotations** button again.

Text Arrow



The Text Arrow tool allows the User to draw an arrow point at a region of interest and add text to the tail end of the arrow. To use the Text Arrow tool, first select the tool, then mouse click and hold where the end of the arrow will be. While still holding the down the mouse button, drag the mouse to where the arrow head will be, then release the mouse button. After releasing the mouse button, a dialog box will pop up asking for the text. Enter the desired text, click **OK**, and the text will appear at the tail end of the arrow. Hotkey: U



Note:

For an arrow without text, do not enter any text into the text field dialog box.


Press **OK** or **Cancel** with the text field empty, and no text will be drawn on the arrow.



Note:

The User may switch the arrow drawing direction in the preferences.

Line

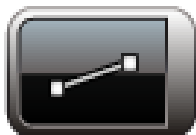

The Line tool allows the user to draw lines on the image. To use the Line tool, first select the tool, then mouse click and hold where the user wants the line to begin, then, while still holding the mouse button, drag out the line. Release the mouse button when the user want to end the line.

Circle



The Circle tool allows the User to draw circles or ellipses on the image. To use the Circle tool, first select the tool, then mouse click and hold where the User want the circle to begin, and while still holding the mouse button, drag the circle out to encompass the region of interest. When finished, release the mouse button to finalize the circle.

Text

The Text tool allows the User to add text notes to the image. To use the Text tool, first select the tool, then mouse click where the text should appear. A dialog box will appear. Enter the desired text and click **OK**. The desired text will appear where the mouse was clicked. Hotkey: T

Overlay

The Overlay tool toggles the amount of information displayed on screen. With this tool the User can display different levels of information about the image. To use the Overlay tool, locate the it in the tool bar, then click the **Overlay** button. There are three different levels of information that can be displayed, so the User can click the Overlay button several times to display different amounts of information.

Filters

Sharpen

The Sharpen tool allows the User to apply a standard image sharpening filter to the image. To use the Sharpen tool, locate it in the tool bar and click it. Depending on the size of the image, it may take a few seconds before results are seen.

Histogram Equalization

The Histogram Equalization tool allows the User to apply a histogram equalization filter to the image. The Histogram equalization tool takes an image that has most of the usable data represented within a close contrast range and "spreads out" this data over the available grayscale values to improve contrast visibility. To use the Histogram Equalization tool, locate the **Histogram Equalization** button in the tool bar and click it. Depending on the size and bit-depth of the image, it may take a few seconds before results are seen. The User may need to re-window/level after applying a histogram equalization.

Undo

The Undo tool reverses the action of the previous Sharpen or Histogram Equalization filter.

Redo



The Redo tool reapplies the last filter that was reverted using the Undo tool.

Reporting Tools



Note:

No annotations or manipulations (W/L, Pan, Zoom, Arrows, etc.) will be included on images in reports. The Reporting Tools are a suite of functions designed to allow the Users to share, view, print and send reports from the Clarity Viewer.

Attach PDF



The Attach PDF tool allows the user to locate a PDF file on the computer and attach it to the study.

Write Report




The Write Report tool opens the Report Writer window. Enter the necessary information by typing in the text field. To create a new line, hit the **Enter** key on the keyboard.

Click **Create** to generate the report. It will be saved alongside the image for viewing.



Note:

Only Users with "doctor" privileges are able to write reports. If your permissions are not set correctly, please contact your System Administrator.

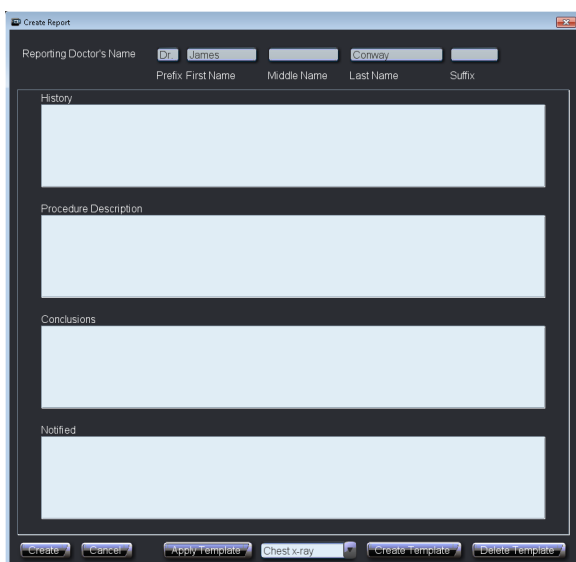


The 'Create Report' dialog box is used for creating a new report. It features a title bar with a close button. Below the title bar, there are input fields for the 'Reporting Doctor's Name', including 'Prefix First Name', 'Middle Name', 'Last Name', and 'Suffix'. The 'Last Name' field is pre-filled with 'Conway'. Below these fields are four large text areas labeled 'History', 'Procedure Description', 'Conclusions', and 'Notified'. At the bottom of the dialog, there is a toolbar with buttons for 'Create', 'Cancel', 'Apply Template', 'Chest x-ray', 'Create Template', and 'Delete Template'.


Clarity Archive only permits one report per a study. To modify a report:

1. Open the patient study.
2. Click the **Write Report** button. The **Create Report Addendum** dialog will pop-up.
3. Select the radio dialog box for either Addendum or Correction, then type the addendum or correction into the text box.
4. Click the **Create** button and the addendum/correction will be appended to the bottom of the report.

Export Report

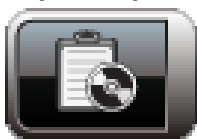

To export a report to the local hard drive for printing, archiving, or other purposes, click the **Export Report** button in the tool bar. A dialog will pop-up asking for the save location. Select a location, then click **Save**. The report will be saved to the location the User chooses. The report can then be opened using any PDF viewer.

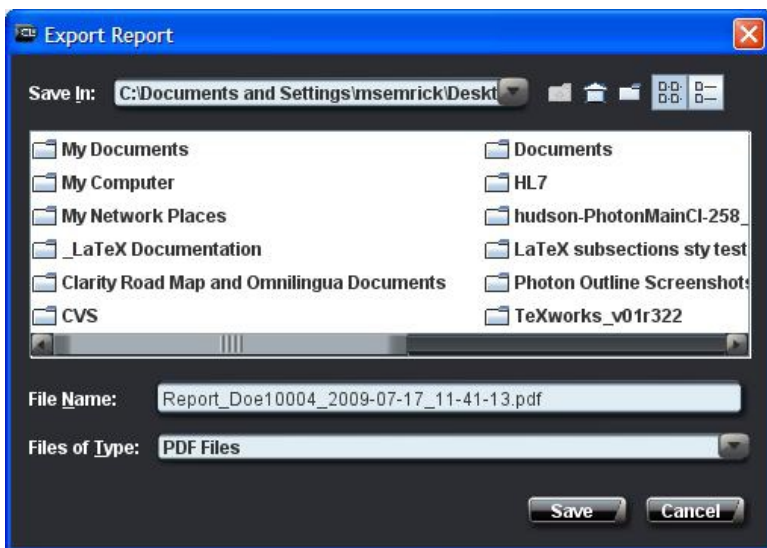

The 'Export Report' dialog box is used for saving a report to a local file. It has a title bar with a close button. The 'Save In' field shows the current directory: 'C:\Documents and Settings\insemrick\Desktop'. Below this is a list of folders and files, including 'My Documents', 'My Computer', 'My Network Places', '_LaTeX Documentation', 'Clarity Road Map and Omnilingua Documents', 'CVS', 'Documents', 'HL7', 'hudson-PhotonMainCI-258_', 'LaTeX subsections sty test', 'Photon Outline Screenshot', and 'TeXworks_v01r322'. The 'File Name' field is pre-filled with 'Report_Doe10004_2009-07-17_11-41-13.pdf'. The 'Files of Type' dropdown is set to 'PDF Files'. At the bottom, there are 'Save' and 'Cancel' buttons.

Save Image



The Save Image tool allows the user to save an image to the local hard drive in the JPEG, GIF, or PNG format.

To export an image to the local hard drive, select the image that wants to be exported and then click the Save Image button in the tool bar. A dialog will pop-up asking for the name of the image and the save location. Select a location, then click Save. The image will be saved to the location the User chooses and it can be opened using any image viewer. By default the file format for the export image is JPEG. The user can export the image as a GIF or PNG format by adding .gif or .png at the end of the image name.



Note:

It is possible to save multiple images on the screen at the same time.

Viewing a Report

Viewing a report is the same as viewing an image. Locate the report in the **Thumbnails** pop-up, then drag the report onto the screen and it will load automatically.



Report Templating

The Report Writer tool includes a templating feature for writing a report.

To apply a template to a report:

1. In the tool bar, click the **Write Report** button.
2. In the drop-down menu next to the Apply Template button, select the desired template.
3. Click the **Apply Template** button.
4. The template will populate the Report text boxes.

Creating a New Report Template

1. In the tool bar, click the **Write Report** button.
2. Fill in the desired fields for the template. The User does not have to fill out all fields. For example, the User could fill in only the **Fax** and **Notified** fields.
3. In the drop-down menu next to the Apply Template button, select **Create New**.
4. Click the **Apply Template** button.
5. Assign a name to the new template, then click **Ok** to save the new template.

Deleting a Report Template

1. In the tool bar, click the **Write Report** button.
2. In the drop-down menu, select the Template to be deleted.
3. Click the **Delete Template** button.
4. The **Delete** dialog will pop-up.
5. Click **OK** to delete the template.

Advanced Tools

The Advanced Toolset allows the user to utilize various tools. These tools are more specific in function than the general tools, and thus may be of interest to certain users.



Note:

The Advanced Toolset is only available to users who have purchased the Advanced Tools package.

To access the Advance Toolset, the user must be in the Study View. Move the mouse to the top left corner of an image. As the mouse hovers over this area, a drop down menu displaying the Advanced toolset will appear. From the menu, select the tool you wish to use.



Pelvic Toolset



The Pelvic Toolset feature contains a number of useful tools related directly to pelvic examinations. The Toolset includes parallel matching lines, perpendicular matching lines, horizontal offset line, cut line, line, point, spine labeling, and measure distance tools.



Parallel Matching Line




The Parallel Matching Line tool allows the user to select an existing line. Once a line is selected, a user can then create new lines that will be parallel to the selected line.

Perpendicular Matching Line



The Perpendicular Matching Line tool allows the user to select an existing line. Once a line is selected, a user can then create new lines that will be perpendicular to the selected line.

Horizontal Offset Line

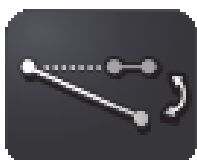


The Horizontal Offset Line tool allows the user to draw a line. Clarity will draw a Horizontal Line at the same height as the first point in the user's line. It will then display the measured distance (MD) in millimeters from the second point in the user's line to the Horizontal line.

Cut Line



Select a line. The Cut Line tool allows the user to create a line that is collinear with the selected line.

Line

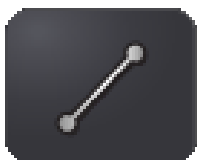

The Line tool allows the user to draw a line on the image.

Point



The Point tool allows the user to create a point on the image.

Spine Labeling



The Spine Labeling tool allows the user to label the vertebrae from any starting point going in either direction.

Measure Distance



Click on a line, point, or annotation. The Measure Distance tool allows the user to determine the distance from that marking to another location on the image.

Curve



The Curve tool allows the user to compare a patient's cervical curve to a healthy reference curve on a lateral cervical image.

To set up the reference curves, go to [Curve Tool](#) on page 27.




Note:

The curve radii should be set to researched values trusted by the physician.



Note:

A physician may select different reference points than those indicated below as long as the points are consistent with the chosen reference curve radius.

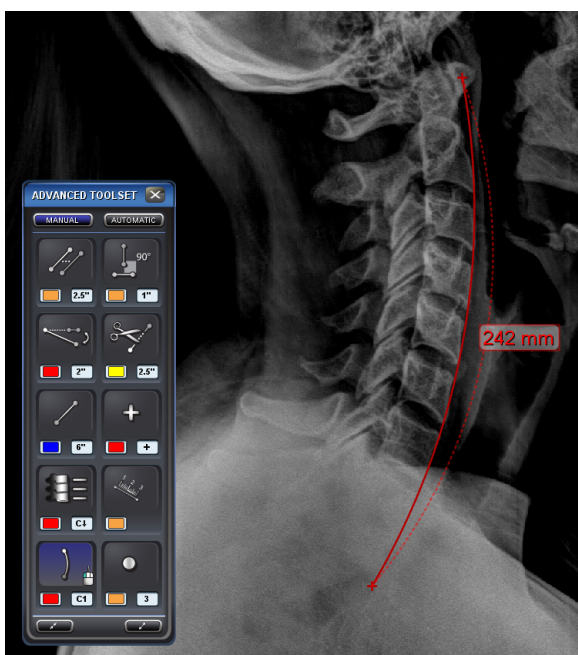

C1 - T2 Cervical Curve

To use the cervical tool:

1. Select the atlantodental interval (ADI) on the C1. This will place a point on the selected area.
2. Place the second point at the bottom of the T2 vertebral body.
3. Place the third point on the edge of the mid vertebral body between C1 and T2 (this will create a curve parallel to the spine).

The completed curve will appear as a solid line and display the radius in mm and degrees. The reference curve will appear as a dashed line.

C2 - C7 Cervical Curve

The C2 - C7 curve should be used when the T1 vertebrae is not visible on the image.

To use the cervical tool for the C2 - C7 curve:

1. Select the bottom of the C2. This will place a point on the selected area.
2. Place the second point at the bottom of the C7 vertebral body.

3. Place the third point on the edge of the mid vertebral body between C2 and C7 (this will create a curve parallel to the spine).

The completed curve will appear as a solid line and display the radius in mm and degrees. The reference curve will appear as a dashed line.

Lumbar Curve

The lumbar curve should be used on a lateral lumbar image.

To use the cervical tool for the lumbar curve:

1. Select the bottom of the T12 cervical body. This will place a point on the selected area.
2. Place the second point at the top of the sacrum S1 cervical body.
3. Place the third point on the edge of the mid-vertebral body between T12 and C7 (this will create a curve parallel to the spine).

The completed curve will appear as a solid line and display the radius in mm and degrees. The reference curve will appear as a dashed line.

Dot



Place a dot on a region of interest by clicking on the desired spot. It is possible to modify the diameter and color of a dot by changing those settings in the toolset.



Note:

The dot will grow or shrink when zooming into or out of the image, respectively.


Other Tools

Horizontal Line




The Horizontal Line tool allows the user to create a horizontal line on the image.

Vertical Line

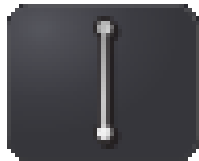


The Vertical Line tool allows the user to create a vertical line on the image.

Parallel Lines

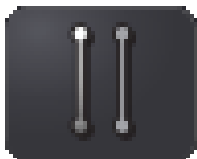

The Parallel Lines tool allows the user to draw a line. The user is then tasked to place a second line, parallel to the first. The tool will then display the distance between the two lines.

Disk Slippage

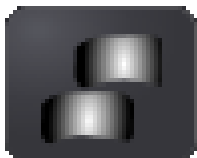

The Disk Slippage tool allows the user to determine the percentage and distance measurements of one disk in relation to another.


Pelvic Incidence Tool



The pelvic incidence tool assists the user to measure the three main pelvic parameters which are pelvic incidence, sacral slope and pelvic tilt.

To use the pelvic incidence tool, first select it from the Advance Toolset menu, then draw a line across the sacral plate by mouse clicking at the start point of the sacral plate, drag the sacral line and mouse clicking at the end point of the sacral plate. Next, drag down the cursor to the center of the femoral head and mouse click to draw a line connecting the midpoint of the sacral plate and the center femoral head. Upon completion of this process, the angles for the sacral slope, the pelvic incidence and the pelvic tilt will be shown at their respective positions.

Distance from Horizontal tool

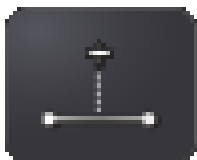


The distance from horizontal tool allows the user to measure the horizontal distance of a point perpendicular from its respective vertical line.

To use the sagittal vertical axis tool, first select it from the Advance Toolset menu, then mouse-click to automatically draw a vertical line on the image. Next, mouse-click at a desired point perpendicular to the vertical line and the horizontal length between the point and the vertical line will be displayed on the image.

By clicking and dragging on the horizontal point, the User can adjust the measurement of the distance from the horizontal line.

Distance from Vertical Tool

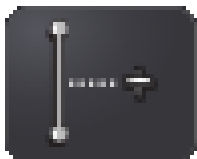

The distance from vertical tool allows the user to measure the vertical distance of a point perpendicular from its respective horizontal line.

To use the distance from vertical tool, first select it from the Advance Toolset menu, then mouse-click to automatically draw a horizontal line on the image. Next, mouse-click at a desired point perpendicular to the horizontal line and the vertical length between the point and the horizontal line will be displayed on the image.

By clicking and dragging on the vertical point, the User can adjust the measurement of the distance from the vertical line.

Angle from Lines Tool



The angle from lines tool allows the user to measure the angle between two lines. To use the angle from lines tool, first select it from the Advanced Toolset menu, then click or drag two lines intersecting each other and an angle will appear at the intersection. The angle can be dragged around the intersection to display the angle of any quadrant. If the lines do not intersect, they will be extended so that they will.

Labels



Predefined labels can quickly be placed on an image with the Labels tool. To use the Labels tool, select the label icon from the Advanced Toolset menu. Left click on the image; a drop-down menu with the predefined labels will appear. Selecting one of the labels will place the label on the image.



Note:

To set up Predefined Labels, go to [Predefined Labels](#) on page 26.

Angle from Horizontal



Given two points, the Angle from Horizontal tool allows the user to determine the angle between the line formed by those two points and the horizontal axis.

To use the tool, select two points on the image. This will create a line between the points and display the angle measurement.

By selecting the angle text and moving it around the image, it is possible to display complementary angles.

Angle from Vertical



Given two points, the Angle from Vertical tool allows the user to determine the angle between the line formed by those two points and the vertical axis.

To use the tool, select two points on the image. This will create a line between the points and display the angle measurement.

By selecting the angle text and moving it around the image, it is possible to display complementary angles.

Circle from Points



The Circle from Points tool lets you create a circle from a series of points.

To use the tool, select at least three points on the image. When finished selecting points, click one of the existing points. Clarity will create a best-fit circle based on the position of the points, displaying the diameter in inches. It is possible to click and drag the points and distance annotation.



Note:

If only one or two points are placed, the tool will be cancelled due to a lack of information.

George's Line



George's Line is a line drawn down the posterior margins of vertebral bodies on the lateral view of the cervical spine. With it, Clarity can generate measurements of vertebral body displacement (mm).

Select the upper point of the cervical vertebra and then the lower point to create a line segment along the posterior. Follow the same steps for the vertebra below it and continue to do so for each vertebra until the desired number of line segments have been created. To stop creating segments, click a location on the image that is at least 12 mm away from the previously placed point. The segments will transform from dotted lines into solid lines.



Note:

If at any point during the segment selection process you switch to another tool or attempt to Window/Level the image, all current segments will be removed. Please be sure to finish segment creation before continuing image manipulation.




Note:

Upon completion of George's Line, Clarity will switch to the Hand tool.

When the George's Line annotation is selected, Clarity will display the lateral offsets for each superior vertebra. The offset distance will be displayed to the side of the offset.

Red segments will connect the vertebra segments together, displaying the lateral offset.

Multi-Angle

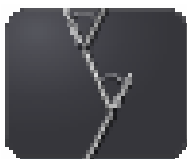

The Multi-Angle tool allows the user to create several connected segments, and will then generate angles between the adjacent segments.

To use the tool, click points to create multiple connected line segments. To end the annotation, click an existing segment and the resulting angles will be displayed. The angle measure values can be dragged to any of the four quadrants, similar to other tools.

Multi-Cobb



The Multi-Cobb tool allows the user to create several segments, and will then generate Cobb angles between sequentially drawn segments.

To use the tool, click points to create multiple line segments. To end the annotation, click an existing segment and the resulting angles will be displayed. The angle value text can be moved just like the regular Cobb tool.

Offsets from Baseline



The Offsets from Baseline tool allows the user to draw an initial baseline, and then create multiple segments, and the tool will then produce angles between the baseline and the segments.

To use the tool, click two points to create the initial baseline segment, and then click to add multiple segments. To end the annotation, click an existing segment, and the resulting angles from baseline will be generated. If a segment does not intersect the baseline, a shaded segment will extend to meet it. The angle measure values can be dragged to any of the four quadrants, similar to other tools.

Perpendicular Offsets from Baseline



The Perpendicular Offsets from Baseline tool allows the user to draw an initial baseline, and then create multiple segments, and the tool will then produce angles between the perpendicular of the baseline and the segments.

To use the tool, click two points to create the initial baseline segment, and then click to add multiple segments. To end the annotation, click an existing segment, and the resulting angles from baseline will be generated. If a segment does not intersect the baseline, a shaded segment will extend to meet it. The angle measure values can be dragged to any of the four quadrants, similar to other tools.

Exporting Studies

Clarity can export studies to any directory, including a CD/DVD drive, USB drive, or network drive.

1. In the Worklist, select the desired patient(s). The User may simultaneously hold down the Ctrl key and left mouse click to select multiple patients.
2. Click the **Export** button.




3. Click **Yes** to retrieve the studies.



Note:

Please be patient, depending on the speed of the network and the number of patients selected, retrieving the studies could take a few minutes.

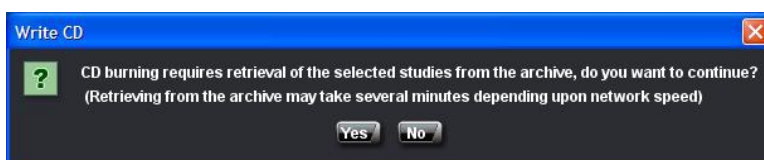

4. Once the studies have been retrieved, the export dialog will appear. Choose an export directory.



Note:


To burn the studies to a CD/DVD, select the CD/DVD drive.



Note:

Clarity Lightviewer is a simplified version of Clarity Viewer that can display saved studies.

If the user wants the lightviewer to be included with the export, mark the **Include Clarity Lightviewer** checkbox.

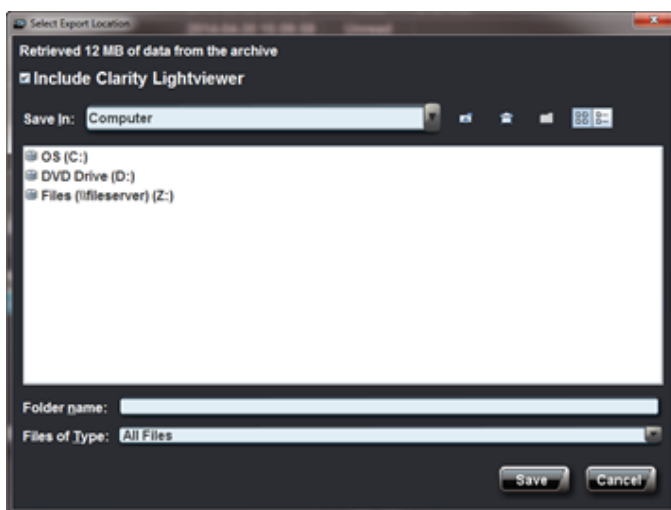

5. Click the **Save** button.



Note:

The **Quit** button will be disabled while exporting the studies.



Note:

Burning a CD/DVD may take a lengthy period of time. Once the disk is done burning, it will automatically eject from the drive.

Saving and Sending a Study


Saving a Study from the Image Viewer window

Clarity is able to save the display state of a study, including all annotations, current window/level settings, rotation position, and flip states. These saved states are called Grayscale Softcopy Presentation States (GSPS), which are part of the DICOM standard. Each saved presentation state is a small file that does not include the actual image; the state references the image.

When in the Image Viewer window, click **Save Study** to save the current presentation state. Multiple presentation states may be saved.



When a study opens, the most recently used presentation state is loaded. To see other presentation states, drag one of the "PR States" thumbnails onto an image. Alternatively, press F3 to bring up the **Presentation State Chooser** dialog. The dialog lists when each presentation state was created and who created each state. Click a row in the table to select a presentation state.

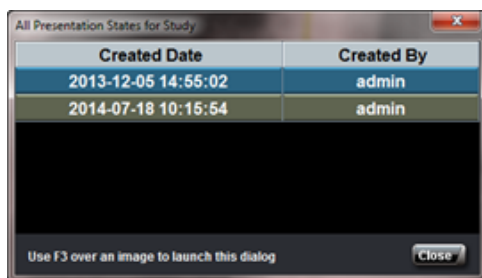

DICOM Sending a Study from the Worklist window

While in the Worklist window, the **Send Study** button is used to DICOM Send to another system's target.

Setting Up a DICOM Target:

1. While in the **Worklist** window, highlight the desired patient(s) to send.
2. Click the **Send Study** button. The **DICOM Send** dialog will pop-up.
3. Select the **Configuration** tab.
4. Enter the new AET Title, IP Address, and Port Number of the DICOM Target.
5. Click the **Save** button. The Clarity Viewer will automatically test the new parameters to make sure there is a valid DICOM connection.
6. If the connection is valid, the **Save AET** dialog will appear. Give the new DICOM target a meaningful name, then click **OK** to save the new target.

DICOM Send:

1. While in the **Worklist** window, highlight the desired patient(s) to send.
2. Click the **Send Study** button. The **DICOM Send** dialog will pop-up.
3. Select the desired profile to send to from the **Profile Name** drop-down menu.
4. Click **Send** to DICOM Send the selected study.

Clarity Viewer Hotkeys

Keyboard Shortcut	Software Function
Escape	Remove all image manipulations (W/L, Zoom, Pan)
F1	Show Online Help Documentation
F2	Brings up CR Overlay Details Dialog
F3	Show Presentation States
F4 (Worklist)	Query/Retrieve Selected Study
F4 (Image Viewer)	Toggle MG CAD Marks
F5 (Worklist)	View Selected Study
F5 (Image Viewer)	Toggle Optional MG CAD Marks
F10	Show Thumbnail Bar
F11	Opens the Pelvic Toolset
Right Arrow	Next Image in a Series
Left Arrow	Previous Image in a Series
Up Arrow	Next Series in Study
Down Arrow	Previous Series in Study
W	Window / Level Tool
Z	Zoom
P	Pan
G	Magnify
D	Drag and Drop
S	Cine Tools Scroll
X	Cine Tools X-Ref
L	Cine Tools Linking
M	Measure
H	Hounsfield Unit
R	Hounsfield Unit Region of Interest
K	Mark Key Image
O	Toggle Overlay Levels
T	Text Annotation
U	Text Arrow Annotation
C	Clear All Annotations
Backspace/Delete	Delete Selected Annotation
Ctrl + A	Select All Annotations

Keyboard Shortcut	Software Function
Ctrl + C	Copy Selected Information to Clipboard
Ctrl + D	Show All DICOM Tags
Ctrl + Alt + Mouse Click	Remove tool from Frequently Used Tools List
Ctrl + Right Arrow	Next Hanging Protocol Stage
Ctrl + Left Arrow	Previous Hanging Protocol Stage
[Rotate Image Counterclockwise by 5 Degrees
]	Rotate Image Clockwise by 5 Degrees

Chapter

4

Mammography

Mammography Features

The mammography features are part of the mammography package.

Linked Zoom

When loading a mammography study or changing a hanging protocol stage, images within the same stage will display at the same initial magnification level.

Chest Wall Adjacency


When a Mammography study opens, the study will display the chest wall lined up against the edge of the screen.



Note:

Clarity will preserve chest wall adjacency while zooming. Certain pan and zoom actions may cause a space to appear between the chest wall and the edge of the screen.

Labeled Priors

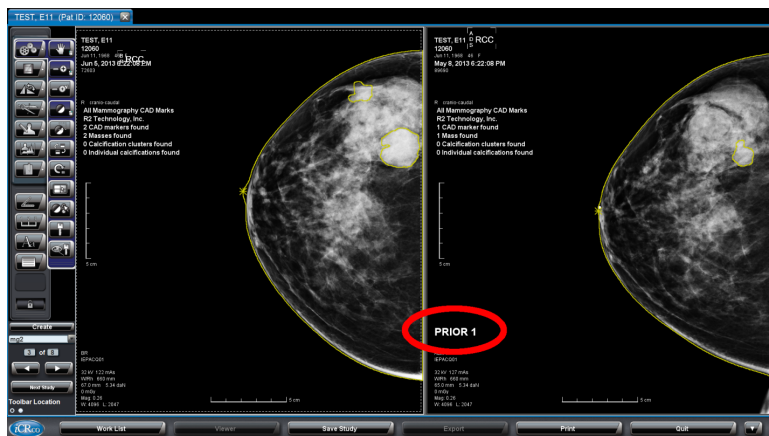

If the overlay level is set to Normal or Full in Configuration, prior mammography images will display a large, white label "PRIOR X", where X is the prior image number.




Note: For example, the most recent prior image will show “PRIOR 1”; similarly the second most recent prior image will show “PRIOR 2”.



Note: For mammography studies, the overlays always appear opposite the anatomy.

Mammography CAD Mark Display

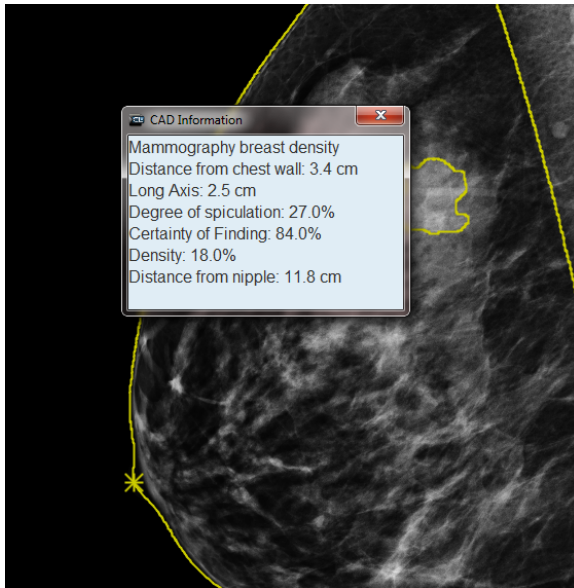

Clarity can be set to display CAD marks if the study was processed with mammography CAD software. A summary of the MG CAD information is shown in the overlay. Select the CAD mark to display detailed information (if available).

If there are multiple MG CAD reports for a study, you may switch between them by dragging the CAD thumbnail onto an image.



Note:

it is possible to toggle CAD marks on and off by hitting the F4 key. The optional CAD marks can be toggled on and off with the F5 key.



Note:

To activate CAD marks and modify the CAD settings, see Section [Mammography CAD](#) on page 27 for more information.

Auto-advance Worklist

The hanging protocol navigation panel includes a “Next Study” button to switch to the next study that has not yet been opened in the worklist.

Hanging Protocols



Note:

This feature requires the Mammography package.

Hanging protocols define the layout of a study and its priors. In Clarity, the user can set up multiple stages, each stage indicating what images are shown on the available monitors.



Note:

It is possible to name the hanging protocols and apply different protocols to the same study.

Manage Hanging Protocols

To manage your hanging protocols, see Section [Hanging Protocols](#) on page 33 for more information. You may view, delete, export, or load your hanging protocols.

To share a hanging protocol, export it to a file. Another user can then log into Clarity and load the protocol from the file.

Hanging Protocol Navigation Pane



You can use the Hanging Protocol Navigation Pane in the toolbar to apply different protocols to the current study. There are options to create or edit a hanging protocol or load a matching protocol from the list.



Note:

When you open a study, the best matching hanging protocol is used.

The pane will list the number of available stages. To move between stages, press the **forward** or **backward** buttons.



Note:

Alternatively, you can also view the next stage by pressing **Ctrl+RightArrow** or view the previous stage by pressing **Ctrl+LeftArrow**.



Note:

A study may not match up with all of the stages of a hanging protocol.

Click **Next Study** to close the current study and load the next, unopened study from the worklist.

Sub Stage Navigation Pane

If more than one image matches the hanging protocol criteria for an image location, it's possible to navigate between these images using sub stages. To bring up the Sub Stage Navigation Pane, hover the mouse icon over the lower, right corner of the image.



Click either the left or right arrows on the pane to shift between matching images.



Note:

If overlays are turned on, the overlay will indicate if sub stages are present.

Creating and Editing a Hanging Protocol

Before creating or editing a hanging protocol, load the study that you want the new hanging protocol to be applied to.

To create a new hanging protocol, or to edit the selected hanging protocol, click **HP Editor**.



Note:

When creating a new hanging protocol, it might be helpful to select the "default" hanging protocol before clicking **HP Editor**.

To edit an existing protocol, choose it from the list of applicable hanging protocols in the Navigation Pane and click **HP Editor**.

Information Tab

The screenshot shows the 'Information' tab of the 'HP Editor' window. The form is divided into two columns. The left column contains fields for 'Name' (mp2m1), 'Description' (mp2m1), 'Level' (MANUFACTURER), and 'Group Name'. The right column contains fields for 'Anatomic Region Sequence' (None), 'Body Part Examined' (None), 'Procedure Code' (None), and 'Modality' (MG). At the bottom of the window, there are 'Cancel' and 'Save' buttons.

In the Information Tab, fill in the basic information relating to the hanging protocol including:

- Name
- Anatomic Region Sequence
- Body Part Examined
- Procedure Code
- Modality




Note:

The information fields will change depending on the study loaded.

Do not overspecify the values as this might prevent the hanging protocol from matching with the desired studies. For example, if you want to create a protocol that works with studies from two different acquisition devices, ensure that both devices assign the tag specified. If their tags differ, set the tag information field to "none".

Layout Tab

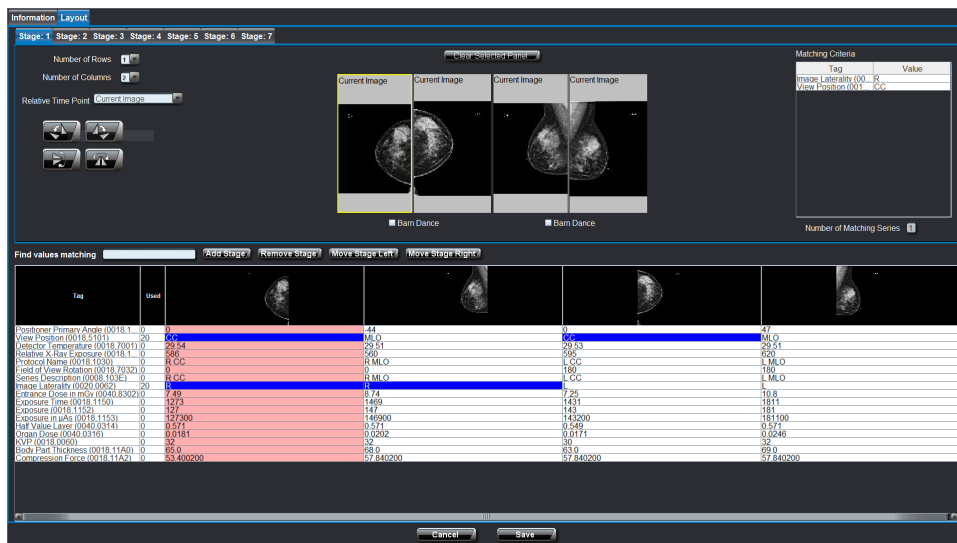

The Layout Tab determines how a hanging protocol organizes and displays images in stages.

Stages

Stages can be added, removed, or shifted to the left or right.



Note:

Clicking **Add stage** will add a stage that follows the currently selected stage. For example, if you selected Stage 3, clicking **Add stage** will insert a stage after Stage 3.

To edit a stage, click the stage's tab. Each stage is comprised of a number of panels, each panel representing a monitor (if you have two monitors, there will be two panels). Each panel is divided into rows and/or columns to create various parts, or locations, each of which can display an image.



Note:

It is possible to adjust the number of rows and columns for each panel.

To edit a location, simply select it. A selected location will be outlined with a yellow border. Adjust the relative time point to determine if the location displays an image from the current study or a prior study.



Note:

The relative time point is displayed on each image.



Note:

Clarity will automatically orient most mammography images. Utilize the **rotate**, **flip**, and **mirror** buttons if further modifications are needed.

Matching Conditions

It's possible to set the matching conditions for where images appear in each panel using the tags table.

To define the conditions for a position, select the position, and then select the desired matching tags.



Note:

It is possible to select multiple tags.

Once a position is specified, the appropriate image will appear in the selected position. The **Matching Criteria** table in the upper right corner will display the matching tags.

Other Features

If the "number of matching series" is greater than one, Clarity will create sub-stages to display all matching images.

The tags table only shows DICOM tags that differ between images in a loaded study. In other words, DICOM tags shared by all of the images within the study will not show up in the table. This allows the user to focus on the tags that differentiate the images.

The **find values matching** field can apply a filter to the tag table so that it only shows tags matching the entered input.



Note:

The filter applies to the tag values and not the tag description.

Selecting the **Barn Dance** checkbox will show all images at the same, relative time point.


OR Relations

OR relations allow a Hanging Protocol to match multiple conditions.


Press the Ctrl button on the keyboard and double-click the right mouse button on a DICOM tag to open a dialog for defining OR relations. For example, clicking the View Position tag will open a dialog asking you to select values for the View Position.



Select **Add Value** to insert an MLO. Click **OK**.

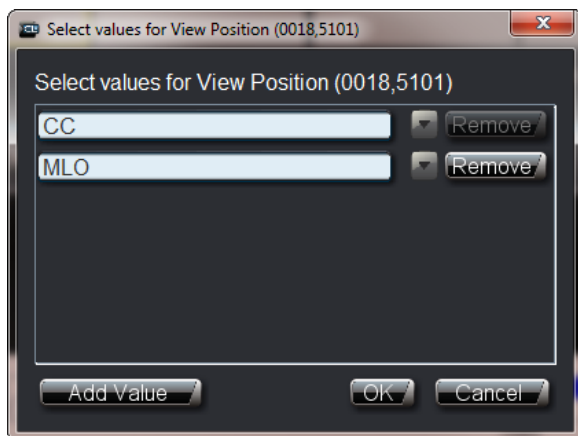

The change will be reflected in the Matching Criteria and in the viewport.

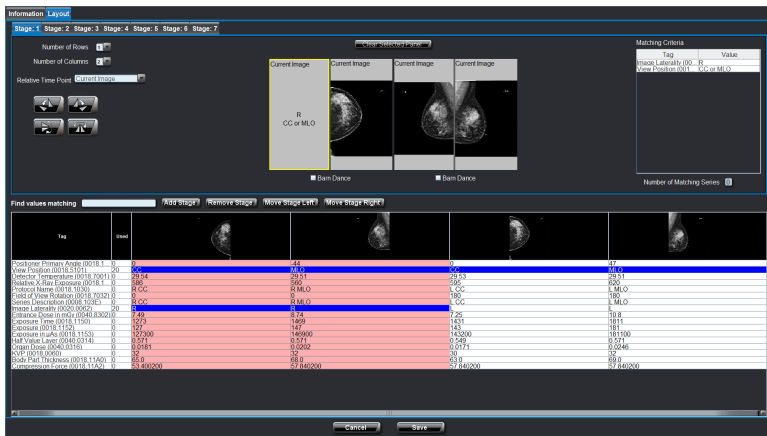

Save Hanging Protocol


Once all desired stages have been defined, the hanging protocol can be saved. The user will be warned if the HP name has already been used.

When all of the desired stages have been defined, save the hanging protocol.



Note:

A warning will appear if there already exists a hanging protocol with the desired name.

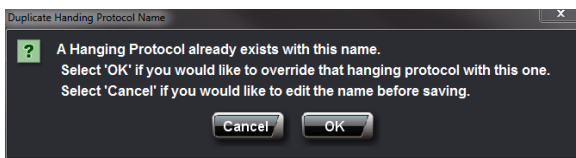

Chapter

5

DICOM Printing

Advanced DICOM Printer

Selecting Studies for Printing



1. In order to launch Print Tool. Select a patient from the worklist.
2. On the **Studies** screen, click the **Print** button. The Print tool will be launched.
3. The program may ask you to allow QPCPrintTool.exe to run. Click **Yes**.




Note:

If the Print Tool is already running, you may receive a warning. You can either add the studies to the program or restart the program.


Drag and Drop Images


To drag images into the print region hold the left mouse button down on one of the image icons on the right hand side and drag it into the preview window. A dragging cursor will appear  as the user drags the image from the image icon window to the preview window. If the user is out of the accepted region for dropping, a cursor  will appear letting the user know that this is not a correct region to drop the image into.

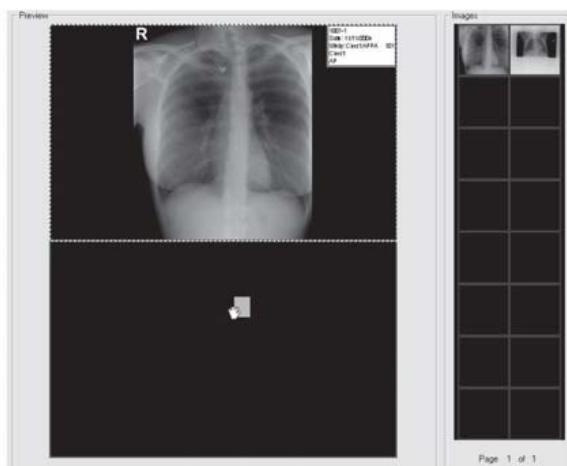


Split the Preview Window into Regions


To split the preview window the user needs to first select the region to split. For example, if there are currently no divisions in the preview window then click anywhere in the preview window and a green line will appear around the window. This will enable the image division buttons. Alternatively, the user can press **Shift** + right mouse click inside the preview window region to split it. A pop-up menu will appear with these division buttons (as the image to the right shows).

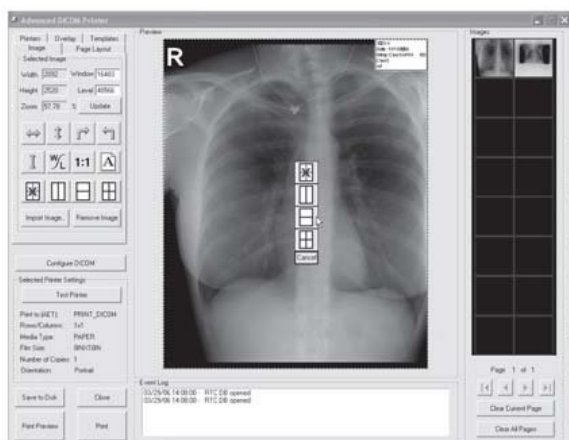


Select one of the image division buttons and the preview window will be divided into regions. To split one of these regions again select the specified region and again select one of the image division buttons. The user can drag and drop images at any stage. If the user has an image in the region that is desired, the interface will automatically move the image into the split regions. To remove a division, click in the region that has been split and click on the no-division  button.

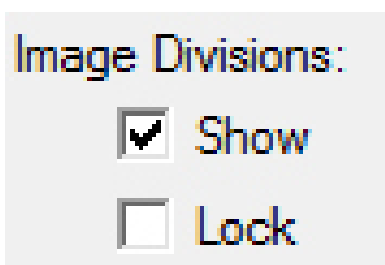

The user can show and hide the image divisions by clicking on the **Page Layout** tab, then click **Show and Lock** under **Image Divisions**. The user can view the current print settings by looking at the left hand side of the **Advanced DICOM Printer** dialog. To change these settings, or to view more settings, click on the **Printers** tab.

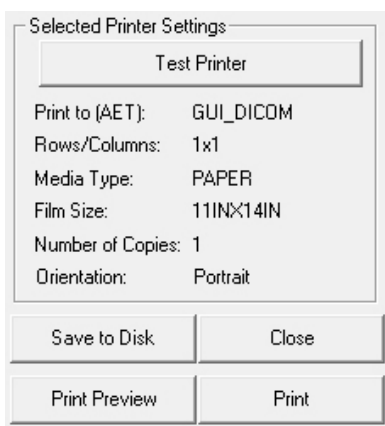

Manipulating Image in the Preview Window

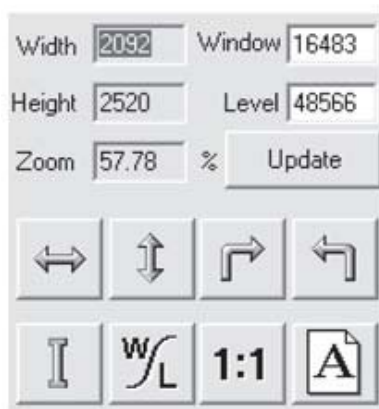

The Advanced DICOM Printer allows the user to flip, rotate, zoom, pan, invert and window/level images. To manipulate the images select the image in the preview window (the selection will be shown by the green box surrounding the image).

The user can flip, rotate and invert by using the buttons on the left hand side of the DICOM Printer dialog.

The user can zoom by using the right mouse button on the selected image.

To pan the image use the left mouse button on the selected image.



Note:

The User can only pan the image when it is zoomed in.


To window/level the user can either use the middle scroll button on the mouse or type the required window/level into the edit boxes on the left hand side of the **Advanced DICOM Printer** dialog. Then press the **Update** button to change the window/level for the image.

The user can also press the **1:1** button to print images at 1:1 resolution.

The user can use the **Orientation** button to change from landscape to portrait view.

Changing Margins

The margin tab permits the user to specify, in English or metric units, the size of the borders on the printed image. The top and bottom borders on the page can be adjusted independently of the borders for the left and the right hand side of the page.

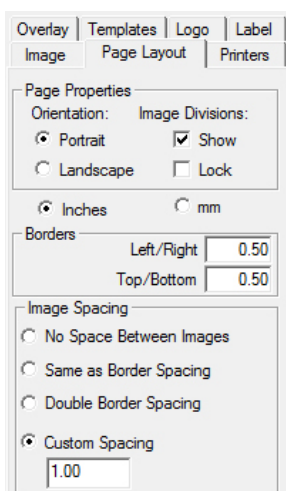

Additionally, the user has the option to control the internal margins, which define the amount of space left between individual frames on a page with multiple images. Internal margins are defined as multiples of the print page border size.


By default, internal margins are the same size as the borders, with image margins on, the left and right hand side of each frame are equal to the left and right hand borders. Margins on the top and bottom of each frame are equal to the top and bottom borders of the printed page.

If the user intends to cut individual frames out of the printed page, it is recommended that internal margins are set to twice that of the page, which will give a uniform border size on the individual image segments. If it is necessary to maximize the amount of space on the printed page, the internal margins can be turned off altogether.

Saving and Loading Print Layouts

The **Templates** tab allows the user to save and load pre-defined print layouts. There are five user defined layouts. Set up the layout on the preview window and then select one of the buttons labeled **Click here to save current layout!** The user will be prompted for a title.

To save more layouts click on the **Save Current Template** button. Here the user can set a file name for the Layout.

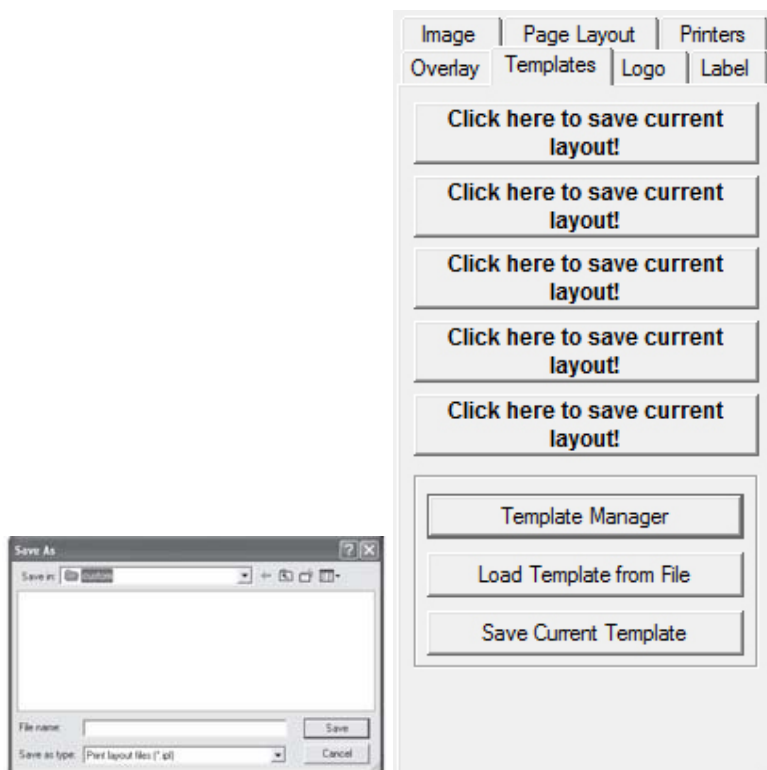

Adding Patient Demographics

Select the **Overlay** tab to bring up the patient demographic layout options.

From here, the User can select:

- Text position (includes top left, top center, top right, bottom left, bottom center, bottom right).
- Text size (includes: small, medium, large).
- Text color (includes: white on black, black on white).

By selecting the **Enable Overlay** check box the user can turn patient demographics overlay on and off for all images. The user can also edit the fields that are available on patient demographics overlay by clicking on the **Edit Fields** button.

- Select a field and determine what kind of a delimiter following the field (includes: none, space, tab, comma, newline, custom).
- Change the position of the fields with the up and down arrows.
- Remove or display a field by selecting the Print Field check.
- Print the title for the field (i.e. the title "Date:" would be printed in front of the date field).


Print Preview

Printing Images to a DICOM Printer

When the user has finished dragging and dropping the desired images onto the preview window and manipulating size, position and window/level (if necessary), then press the **Print** button in the bottom left hand corner of the **DICOM Printer** dialog. Please ensure all the printer settings are correct first.

Patient Demographics for Icons

When the cursor is over the icon bank, Advanced DICOM Print will display the demographics for the image.

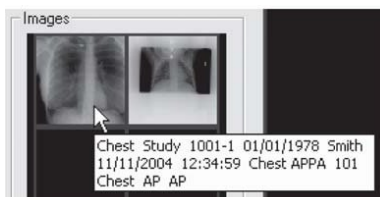

Multiple Icon Pages

Advanced DICOM Print has an unlimited number of icons that can be loaded. However, the PC's memory is a restriction.

To clear the current page of icons, click on **Clear Current Page** button. To clear all the pages of icons and preview window, click on **Clear All Pages** button.

To move between pages and move to first and last pages click on the arrow buttons.



Printing to a Paper Printer


1. After QPC Print Tool has launched, go to the DICOM Application Print Server window.
2. Click the Printer button.
3. Select the Paper Printer you wish to use from the drop-down menu and click **OK**.




Note:

QPC Print Tool can only use a Paper Printer that is a part of the user's network.

- Click the Settings button.

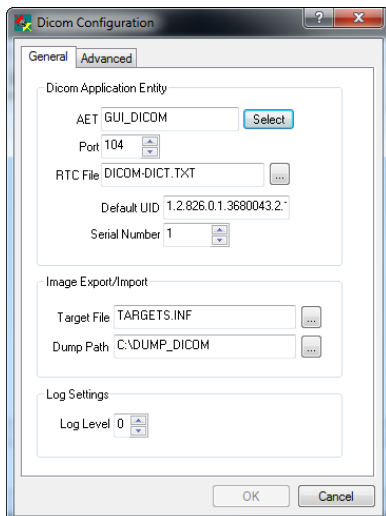


- Click the **Select** button located to the right of the AET field.
- If a Paper Printer is already set up, the user can select one of the existing settings. Else, they should click **Add New**

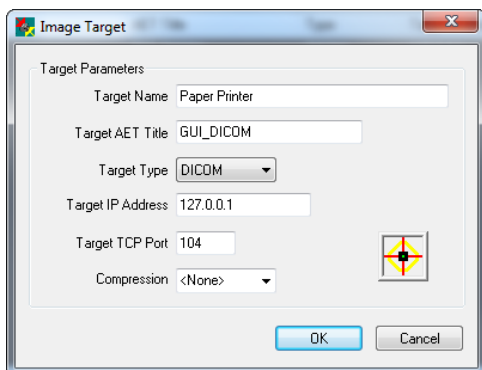


Enter the following information:

- Target Name: User can set the name of the printer
- Target AET Title: GUI_DICOM
- Target IP Address: 127.0.0.1
- Target TCP Port: 104


Click **OK**. Select the new settings and click **OK**.

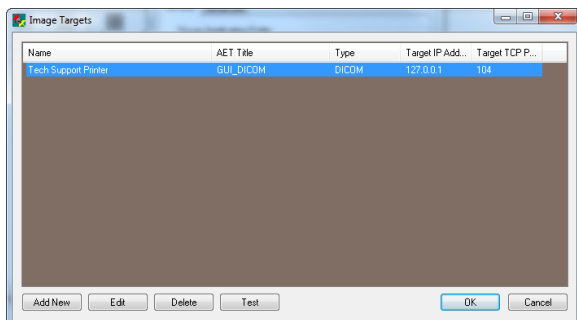

- In the Dicom Configuration window, set the Serial Number to "1", and then click **OK**.

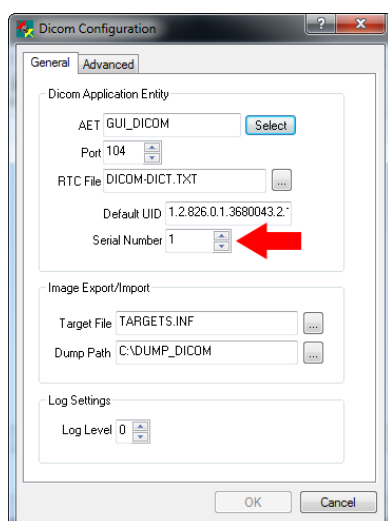

8. Go to the Advanced DICOM Printer window and select the Printers tab.

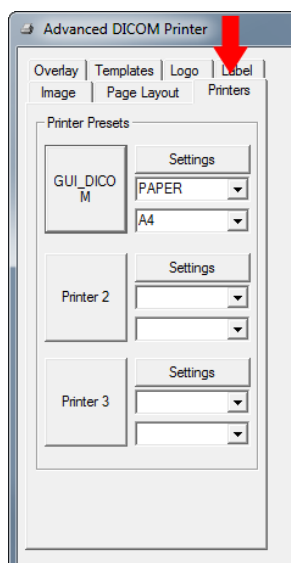


9. In the Printer Presets section, click the **Settings** button of one of the preset printers. This will open the DICOM Print Settings window.




Note:

For this example, the settings button of Printer 1 was selected.

10. In the DICOM Print Settings window, select the Printer tab.

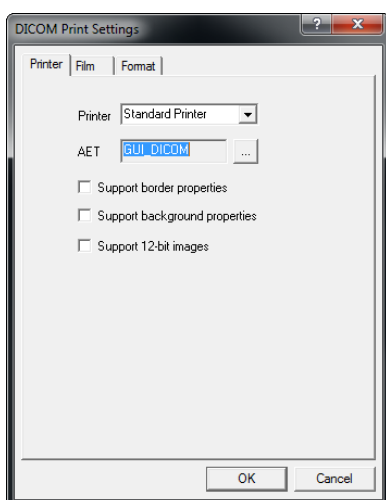

11. From the Printer drop-down menu, select the "Standard Printer" option.
12. Click the ... button to the right of the AET field to set the AET. Select the AET settings you entered earlier in the DICOM Application Print Server window. Click **OK**.
13. Select the Film tab.

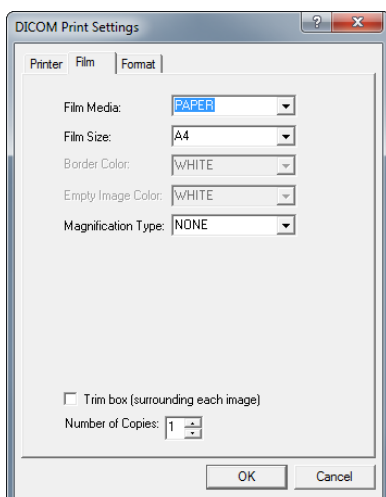

14. Change the Film Media setting to "PAPER" and the Film Size setting to "A4".



Note:


A4 size is equivalent to 8.5 x 11 inch paper.




Note:

It is also possible to print on 11 x 17 inch paper by selecting the "11INX17IN" size.

15. Click **OK** to close the DICOM Print Settings window.
16. Select the **Configure DICOM** button to open the Dicom Configuration windows.

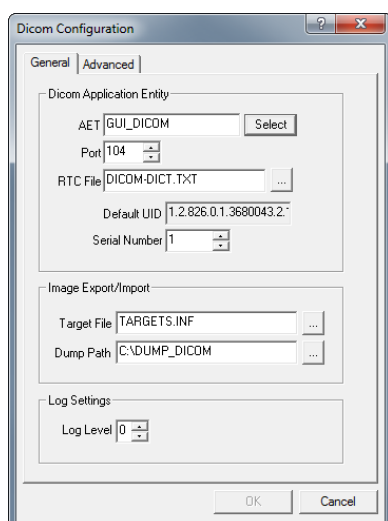

17. In the General tab, click the **Select** button to the right of the AET field. Select the AET settings you entered earlier in the DICOM Application Print Server window. Click **OK**.
18. Click **OK** to close the Dicom Configuration window.
19. Select the GUI_DICOM printer preset in the Printers tab.

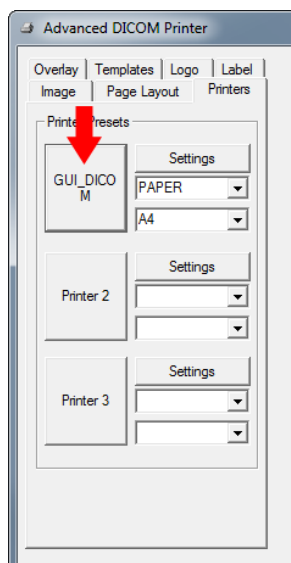

20. Select the **Test Printer** button. If the settings are correct, the program will state: "Successfully associated a connection with GUI_DICOM". Click **OK**.
21. Click **Print** to print the image using a paper printer.



Note:

Be sure an image is selected for printing. To select an image for printing, click and drag the image from the Images field on the right to the black Preview field in the center of the Advanced DICOM Printer window.



Note:

If the Printer Test was successful, but the paper printing failed, the network settings might need to be modified. Contact your facility's IT personnel.

DICOM Print Troubleshooting for Windows

Multiple Instances

Ensure that there is only one instance of QPC Print Tool running.

1. Open Task Manager by clicking on **Start Menu**, then **Run**.
2. Type in "taskmgr" and click **OK**.
3. Close QPC Print Tool.
4. Under the Processes tab, check if DCM.exe, DICOMMultiPrinter.exe, or AppPrintSCP.exe. is running. If so, close them by clicking **End Process** for each of them.
5. Reopen QPC Print Tool.

Target Information

Check the target information for both the Advanced DICOM Print and DICOM Application Printer Server.

1. Select Advanced DICOM Print and click on Configure DICOM button, the DICOM Configuration dialog will appear.
2. Click on Select to see full details of this target.
3. If Advanced DICOM Print has not been entered as an item in the targets list then fill in the relevant settings.



Note:

The target details for this application are usually: AET: ADV_PRINT_DICOM, PORT: 4008, and IP: 127.0.0.1

4. Select this new target and click **OK**.
5. Click **OK** on the DICOM Configuration to save settings.
6. Close QPC Print Tool and reopen Select DICOM Application Printer Server.
7. Click on the Settings button. The DICOM Configuration dialog will appear.
8. Click **Select** to see full details of this target.
9. If DICOM Application Printer Server has not been entered as an item in the targets list then fill in the relevant settings.



Note:

The target details for this application are usually: AET: PRINT_DICOM, PORT: 3008, and IP: 127.0.0.1

10. Select this new target and click **OK**.
11. Close QPC Print Tool and then reopen the program.
12. Go to Advanced DICOM Print and test the printer.



Note:

This assumes that the user has the DICOM_PRINT target selected as the default printer to print to.

To set the DICOM_Print target as the default printer, select the Printer tab in ADVANCED DICOM PRINT and select the DICOM_PRINT button.

13. If this fails, recheck that multiple instances of QPC Print Tool are not open in Task Manager.

Properties

Ensure that all selected and unselected options are correct.

- If the printer does not support border properties, deselect this option.
- If the printer does not support background properties, deselect this option.
- If the printer does not support 12-bit images, deselect this option.

Print Options

Check the print options to ensure all settings are correct.

- If printing to a paper printer, the user cannot have BLUE_FILM selected (unless it uses BLUE_FILM). The PAPER option should be selected.
- If the wrong paper size is selected, the DICOM Application Printer Server will notify you that the operation has failed. The message log will say incorrect paper size.



Note:

The default paper size for a paper printer is 8INx12IN.


Configuring a Printer with the QPC Print Tool

The following section demonstrates how to setup a DICOM film printer with the QPC Print Tool.

1. Open the QPC Print Tool.
2. On the Printers tab, select the **Settings** button.
The DICOM Print Settings window will appear.
3. Select the printer make and model from the **Printer** drop-down menu. If the make and model of your printer is not available, select **Standard Printer**.
4. Select the "..." next to AET.
The Image Targets window will appear.
5. In the Image Targets window, select **Add New**.
6. Enter the target system's AET, IP address, and port number. Ensure the compression is set to "none" and the Target Type is "DICOM".
Click **OK**.
7. In the Image Targets window, select the target you just created, then select the **OK** button.
In the DICOM Print Settings window, select the **OK** button.
8. In the Advanced DICOM Printer window, select the **Printer 1** button (or the applicable printer), then select the **Test Printer** button to test the association to the printer.

Clearing the Java Cache

Clearing the Java Cache will remove Java Web Start programs.

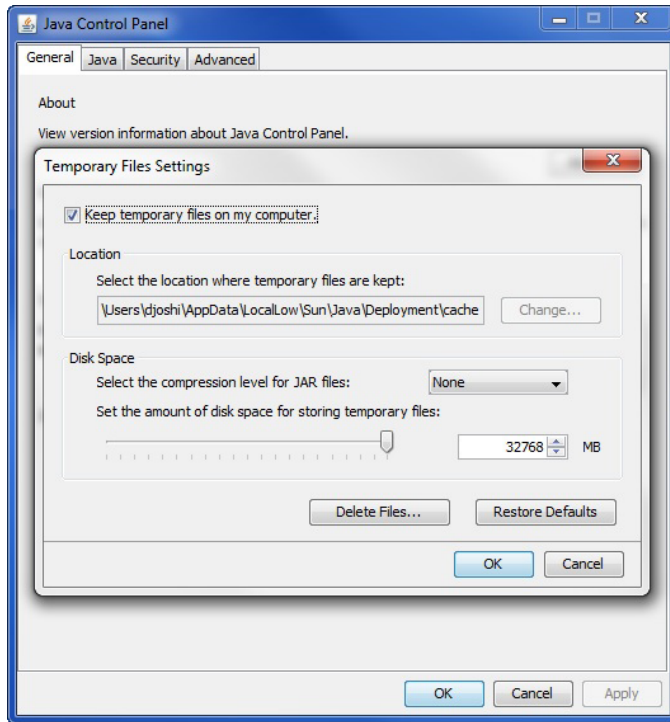

1. Go to **Start > Control Panel > Java**.
2. Locate the **Temporary Internet Files** section. Click the **Settings** button.
3. Click the **Delete Files** button.
4. Make sure **Applications and Applets** and **Trace and Log Files** are both checked, then click **OK**.
The Java Cache is cleared.

Setting the Display Back to Default Using Regedit


If the monitors are not displaying properly, the User should reset them to their default state.



Warning:


Improper editing of the Windows Registry may result in a non-functioning system and/or loss of data. Making regular data back-ups is the only way to ensure data integrity. iCRco assumes no liability for lost or corrupted data while following these instructions.

1. Go to **Start > Run**.

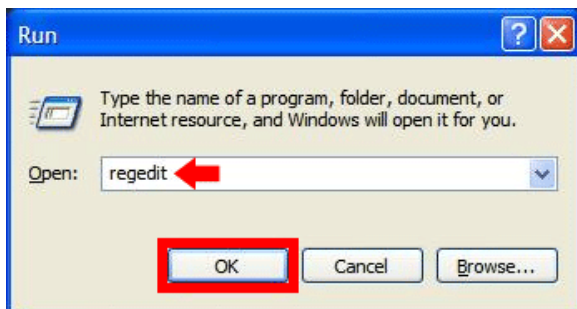


2. Regedit will open.


3. In the file tree on the right side of Regedit, go to **HKEY_CURRENT_USER > Software > JavaSoft > Prefs > com > icrco > ui > prefs > sys**.

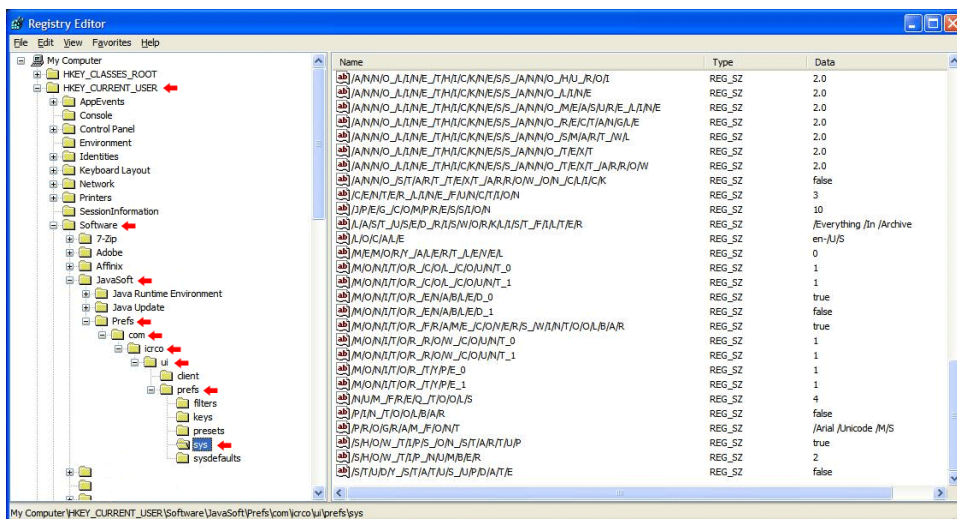

4. Locate the string `/M/O/N/I/T/O/R_/E/N/A/B/L/E/D_0`, double click the string and set the Value Data to true.

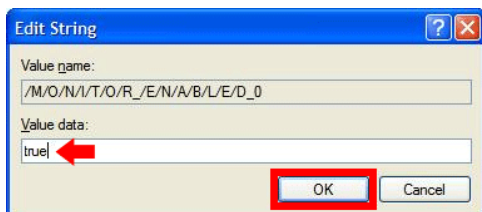

5. Any subsequent monitors attached to the system and detected by Clarity will have a string like: `/M/O/N/I/T/O/R_/E/N/A/B/L/E/D_1`. Locate the rest of the monitor strings and set their **Value data** to false. Click **Ok** to save the changes.

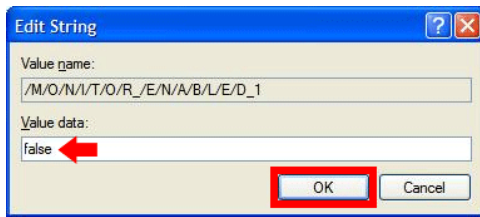

6. On the next launch of the Clarity Viewer, the primary Windows monitor should be the only monitor Clarity uses. From this point, the User may log into Clarity and re-configure the monitors, as described in Section [Monitor](#) on page 23.